## ENGLISH

### INTENDED USE /INDICATIONS:

OA knee bracing products are intended to provide relief for medial and lateral compartment Osteoarthritis.

### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Not Applicable.

### ⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.
- If you experience increased pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

- For sliding activities, it is recommended that a slide guard (pn.22000,22001) be worn with the brace. Wearing a slide guard is intended to minimize impact to the brace. The slide guard may not be effective in all situations.
- Care and cleaning of this product is essential to its continuing strength and performance. Follow removal and cleaning process described in instructions.
- When using braces bilaterally, modify activity as necessary to keep medical hinges from contacting each other.
- When using a Fusion brace equipped with the OA hinge it is important to keep dirt and grit out of the hinge mechanism to maintain proper hinge mobility and brace function.
- For single patient use only.

### INSTRUCTIONS FOR USE:

#### APPLY BRACE TO LEG

- Sit** in a chair with your foot flat on the floor, and the knee bent at around 45 degrees. Unfasten all straps.
- Slide** the brace onto the leg positioning the center of the hinge with the top of your kneecap.
- Press** the shin cuff down onto the leg and fasten strap #1 behind your calf.
- Fasten** strap #2 in the back above the knee.
- Push** the thigh cuff down against the leg and fasten strap #3.
- Fasten** strap #4 in front below the knee.
- Fasten** strap #5 in the back below the knee.
- Stand** up and fasten strap #6 in front above the knee.
- For PCL indications follow the strapping sequence in figure E. and instructions above.**
- To remove** the brace, simply loosen straps 4 & 2 (ACL version) or straps 5 & 6 (PCL version). Then, unfasten, and remove all the straps behind the knee and remove the brace. Fold the hook/loop tabs back against the straps. This will make it easier to re-apply the brace and allow the hinges to be placed in the proper posterior position.

#### FITTING TIPS

- Tighten straps tight but comfortably, straps may need to be loosened and re-tightened to get a comfortable fit.
- Make sure the pads on the straps are centered to provide comfortable contact for the straps on the skin.
- Strap length may be adjusted by loosening the end of the strap attached to the brace, and either lengthening or shortening the strap.

#### HINGE RANGE OF MOTION ADJUSTMENT

(Figure 1) Brace is shipped with a 10 degree extension stop installed. Additional flexion and extension stops are included.

To install a hinge stop:

- Remove the condyle pad
- Using a screwdriver remove the hinge screws
- Remove any stops already installed
- Insert required stops
- Replace screw and position the condyle pad in its correct position

#### OSTEOARTHRITIS (OA) HINGE INSTRUCTIONS (Figure 2)

With knee in a bent or straight position:

- Turn the dial clockwise to increase the outside load on the knee.
- Turn the dial counter-clockwise to decrease the outside load on the knee
- The load is located in the window on the dial and ranges from 0-10

#### LATERAL OSTEOARTHRITIS (OA) HINGE INSTRUCTIONS (Figure 2)

- Turn the dial clockwise to increase the inside load of the knee. Turn the dial counter-clockwise to decrease the inside load on the knee each turn is .5mm.

#### USE AND CARE:

Straps must be removed and cleaned regularly depending upon your activity level and perspiration. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry. Occasionally apply a dry lubricant to hinge especially after use in water.

#### MATERIAL CONTENT:

Aluminum, EVA, Nylon, Polyurethane, Stainless Steel.

#### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

#### DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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