ENGLISH

INTENDED USE/INDICATIONS:

Foot and ankle products provide overall support, compression and/or protection to the lower leg including the ankle, and foot after an injury or surgery. Indicated for high ankle sprains, sub-acute ankle sprains, chronic ankle instabilities, and prophylactic use.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient’s caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Do not contact or place a hot brace against the skin immediately after heating.
- Do not exceed the recommended temperature and time for heating the brace.
- Only use an infrared oven or conventional hair dryer to heat the brace. Never use a convection oven, conventional oven, water bath, or microwave as a heat source.
- While the Ultra High-5 brace may be used for injury prevention, no ankle brace can prevent all ankle injuries and no guarantee is made either express or implied that any injury will be prevented by this brace.
- Wear an absorbent full tall sock when using this product.

INSTRUCTIONS FOR USE:

BREAK-IN PERIOD

New shoes combined with a new ankle brace require an even longer break-in period (usually 6-8 hours). Loosen the shoelaces to reduce friction until the brace and the shoe break-in. Wear an additional full tall sock to help reduce friction.

CUSTOM ORTHOTIC OR ARCH SUPPORT

Place the bottom of the brace beneath the orthotic or arch support.

When using an infrared oven, the recommended temperature range for heating the brace at is 130-149°F (54-65°C) for 3 minutes. If using a conventional hair dryer, place the brace on a flat surface and heat the brace for 3 minutes or until all sides feel warm. Take precaution after both heating methods and let the brace cool for at least 20 seconds before handling the brace. Remember to always wear an absorbent full tall sock.

- If your shoe has a removable insole, place the Ultra High-5 beneath it.
- Place your foot inside the shoe. Then slide the bottom of the brace toward the back of the shoe as far as it will go.
- Secure the strap firmly and tie shoe laces.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Ethylene Vinyl Acetate (EVA), Polyester, Polypropylene, Stainless Steel

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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