Achilles Wedges

FITTING INSTRUCTIONS

EC REP

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

AW-1.00435 REV D 0113

⚠️ WARNINGS

- WARNING:** CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.
- WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.
- WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
- WARNING:** TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.
- WARNING:** DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING WALKER BOOT.
- WARNING:** ONLY USE ACHILLES WEDGES IN CLOSED HEEL WALKING BOOTS. IF THE WALKING BOOT CONTAINS A REMOVABLE HEEL CLIP, MAKE SURE THE HEEL CLIP IS INSERTED INSIDE THE HEEL OF THE WALKING BOOT.
- CAUTION:** FOR SINGLE PATIENT USE ONLY.

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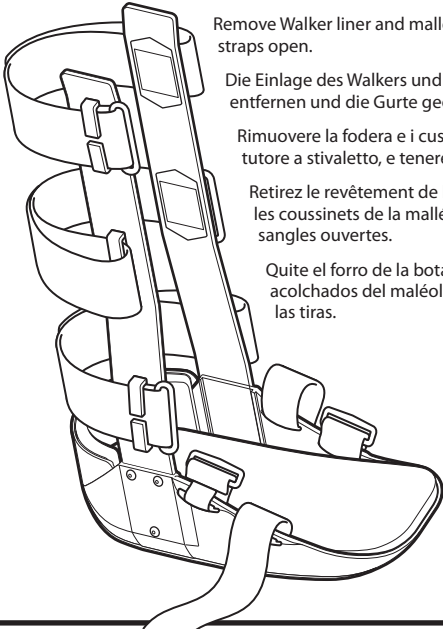
1

Remove Walker liner and malleoli pads, keep the straps open.

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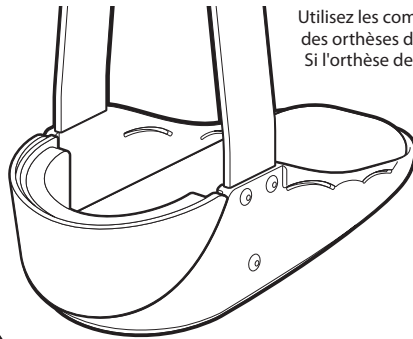
2

Only use Achilles Wedges in closed heel walking boots. If the walking boot contains a removable heel clip, make sure the heel clip is inserted inside the heel of the walking boot.

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3

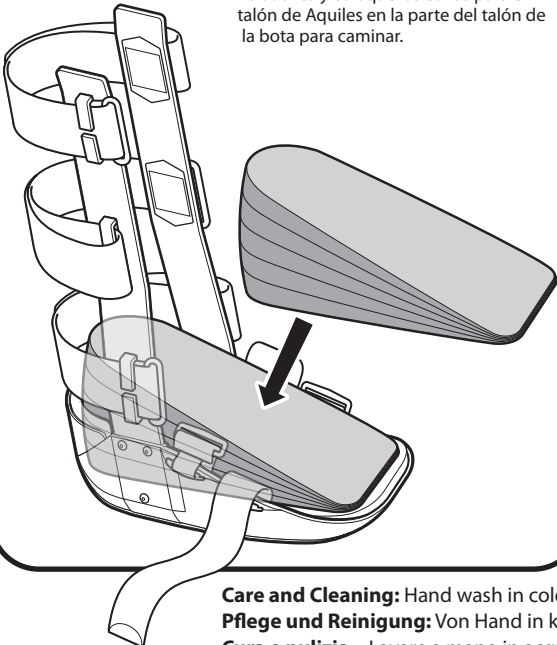
Insert and position the Achilles wedges towards the heel of the walking boot.

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4

Properly installed wedges. Extension straps can be added if the standard straps are too short.

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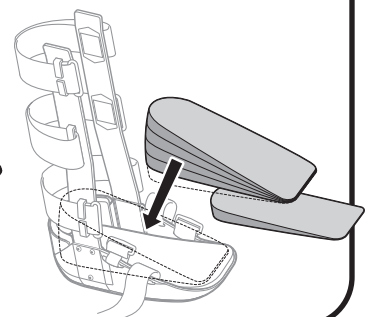
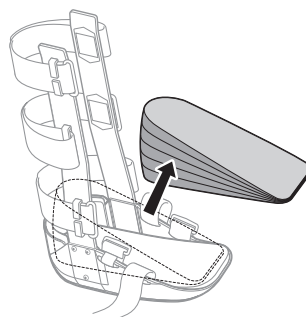
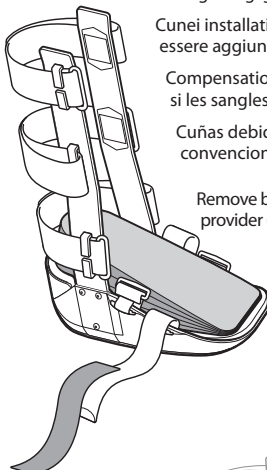
Remove bottom layer of wedges to obtain proper angle as indicated by health care provider (each layer is 5°).

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Care and Cleaning: Hand wash in cold water with mild detergent, rinse, air dry.

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