ENGLISH

INTENDED USE/INDICATIONS:

The intended use for Walker, Ankle, and Foot bracing products is to provide support, relief, or protection to the lower leg, ankle, or foot. The Achilles wedge is used in conjunction with a walking boot to provide controlled heel elevation for patients recovering from Achilles tendon injury.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read use/care instructions and warnings prior to use.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program.
- If you experience increased pain, swelling, or any adverse reactions while using this product, immediately consult your medical professional.
- Take appropriate precautions when walking on slippery surfaces, uneven terrain, or any other property condition where the exercise of caution appears necessary.
- Do not operate a motor vehicle or human powered vehicle while wearing walker boot.
- Only use Achilles wedges in closed heel walking boots. If the walking boot contains a removable heel clip, make sure the heel clip is inserted inside the heel of the walking boot.
- For single patient use only.

INSTRUCTIONS FOR USE:

- Remove Walker liner and malleoli pads, keep the straps open.
- Only use Achilles wedges in closed heel walking boots. If the walking boot contains a removable heel clip, make sure the heel clip is inserted inside the heel of the walking boot.
- Insert and position the Achilles wedges towards the heel of the walking boot.
- Properly installed wedges. Extension straps can be added if the standard straps are too short.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Ethylene-vinyl acetate (EVA), Nylon, Polyester, Polypropylene

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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