1

ENGLISH

INTENDED USE/INDICATIONS:

The intended use for Walker, Ankle, and Foot bracing products is to provide support, relief, or protection to the lower leg, ankle, or foot.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Follow application and care process described in instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

- Open the ankle stirrup so inner liner is facing upward.
- Position the heel evenly on the center pad.
- Center side shells along ankle and leg.
- Apply pressure to sides of foam liner for even flow of pressure.
- Adjust vertical side straps and secure to sides of the stirrup.

USE AND CARE:

Outer shell can be wiped with mild soap and water. Fasten hook and loop straps. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:


Cotton, Nylon, Polyethylene

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

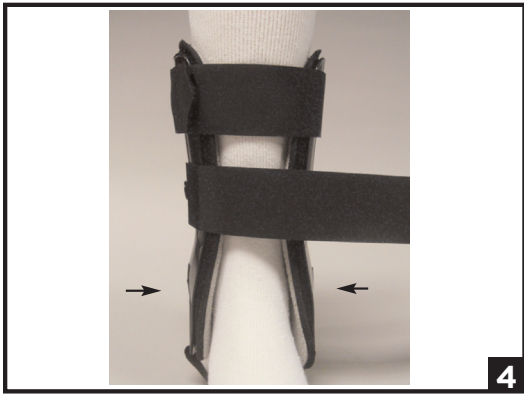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