INDICATIONS: For non-surgical or post-operative support and immobilization of the injured shoulder when varying degrees of internal rotation, external rotation and abduction are required. Positions include gunslinger, neutral plane, and statue of liberty.

WARNING: For Single Patient Use Only.

CAUTION: This device is not intended for reuse on a second patient or for extended use beyond 180 days.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device will not prevent or reduce

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all injuries.

CAUTION: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CAUTION: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time.

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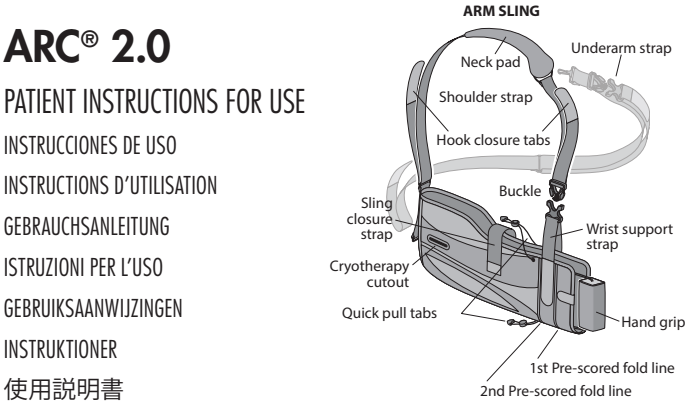
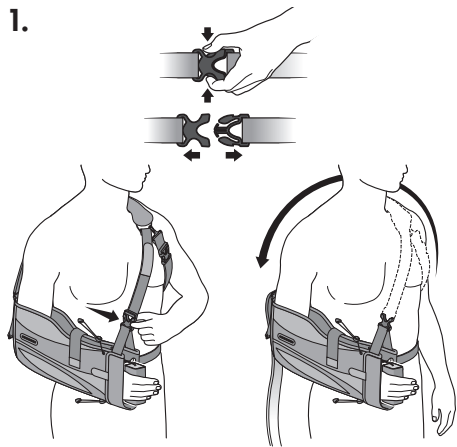
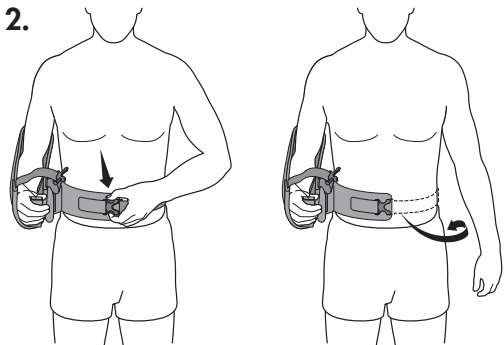
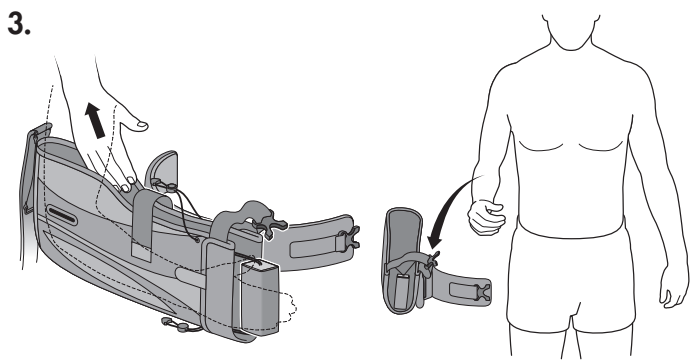
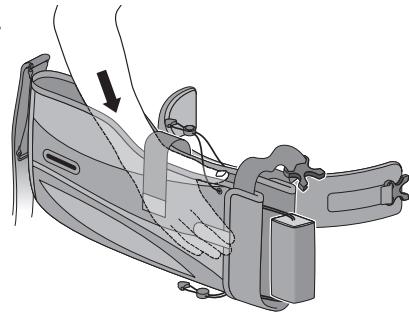
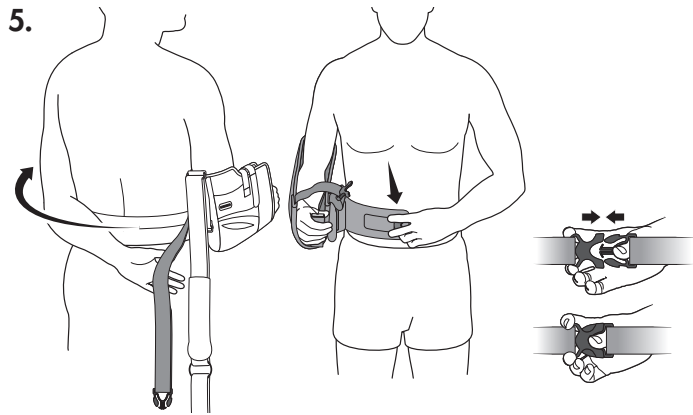
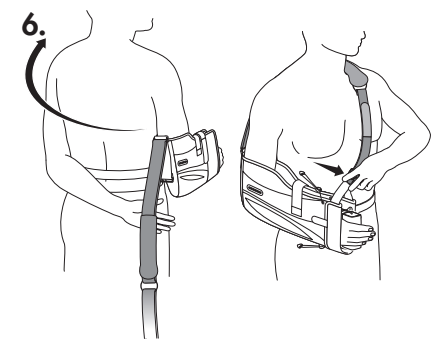
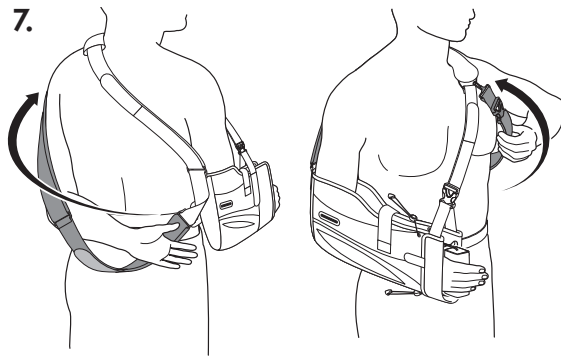
(Right Shoulder Shown)

TO REMOVE THE ARC 2.0

1. Release the buckle of the shoulder strap, and let it slide off the back shoulder. If the underarm strap is being used, release the buckle on the underarm strap, leaving it clipped to the ring inside the neck pad.

2. Release the buckle of the ARC torso strap, and let it fall behind the back. Leave the ARC attached to the arm sling.

3. Remove the sling and ARC as one unit from the affected arm. Leave the sling and ARC attached to each other for re-application later.

TO RE-APPLY THE ARC 2.0

4. Slip affected arm into the arm sling. The arm sling and ARC should still be attached to one another.

5. With unaffected arm, reach behind your back and pull the ARC torso strap around to the front and attach the buckle at the front.

6. With the unaffected arm, reach behind your back and pull the shoulder strap around to the front and insert the buckle into the wrist support strap of the arm sling.

TO RE-APPLY THE ARC 2.0 WITH UNDERARM STRAP ATTACHED

7. Slip the unaffected arm between the shoulder strap and remaining strap. Insert the remaining buckle to secure.

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© 2023 Breg, Inc. All rights reserved.