ENGLISH

INTENDED USE/INDICATIONS:

Soft knee bracing products are designed to provide control and support for ligament injuries and instabilities.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient’s caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Follow application and care process described in instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

- 1. Apply to Knee**
Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the knee so that the opening at the front surrounds the knee cap.
 - a. Buttress shown in multiple positions.
- 2. Secure Straps**
Starting with the lower strap, feed straps through the D-rings and secure. Fit should be snug but not too tight so as to restrict circulation.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Ethylene Vinyl Acetate (EVA), Neoprene, Nylon, Stainless Steel

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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