1

Please select the correct size as the collar comes in different heights and circumferences. Apply collar piece "Back" to the back of patient's neck. Height – measure bottom of chin to top of sternum.

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2

Apply front half of collar with chin secured in recess. Center collar to ensure neutral alignment. Front half overlaps back half.

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3

With the hook and loop fasteners tighten collar with bi-lateral adjustments. Center collar with arrow pointing up. Adjust if necessary.

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© 2023 Breg, Inc.
All rights reserved.
AW-1.94000
REV. E 10/23

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany

WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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