**INDICATIONS:** For patellofemoral disorders, including lateral patellar subluxation, dislocations, lateral patellofemoral malalignment, and mild MCL and LCL support.

**WARNING:** Federal Law Restricts this Device to Sale by or on the Order of a Licensed Health Care Practitioner.

**WARNING:** For Single Patient Use Only.

**CAUTION:** Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

**CAUTION:** This device will not prevent or reduce all injuries.

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**CAUTION:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

**CAUTION:** This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

**CAUTION:** Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time.

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