## WARNINGS

**WARNING:** Remove before walking. Walking while wearing the Dorsal Night Splint could result in slipping and falling.

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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REV. E 06/24

**1**

Remove the Dorsal Night Splint from bag and open hook closures.

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**2**

Slide foot and heel into elastic piece and tighten the ankle straps and the toe straps. Readjust if necessary for comfort.

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**3**

Wrap the calf strap around calf, and adjust for comfort if necessary.

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#### CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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#### NOTICE TO USERS

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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© 2024 Breg, Inc. All rights reserved.