FITTING INSTRUCTIONS

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MISE EN PLACE

INSTRUCCIONES DE AJUSTE

E/U authorized representative
MDSG GmbH
Schiffgraben 41
D-30175 Hannover
Germany

Scan QR Code
for INSTR. VIDEO

breg.com/fsk

Breg, Inc.
2382 Faraday Avenue, Suite 300
Carlsbad, CA 92008 USA
P: 800-321-0607
760-795-5440
F: 800-329-2734
www.breg.com
©2023 Breg, Inc.

AW 1.01135 REV B 1023

WARNINGS

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS.

WARNING: IF YOU EXPERIENCE, OR HAVE AN INCREASE IN PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: FOLLOW APPLICATION AND CARE PROCESS DESCRIBED IN INSTRUCTIONS.

CAUTION: FOR SINGLE PATIENT USE ONLY.

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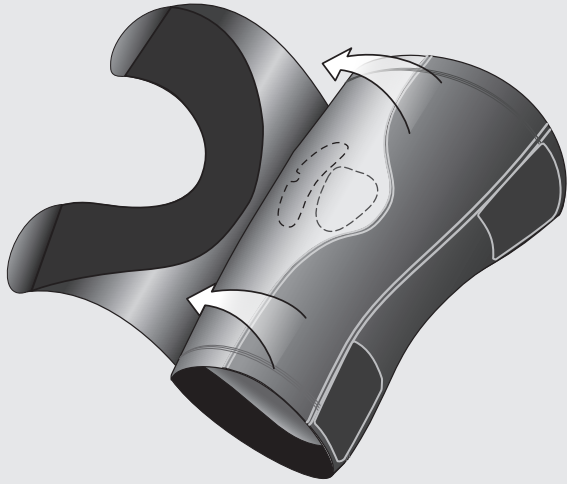
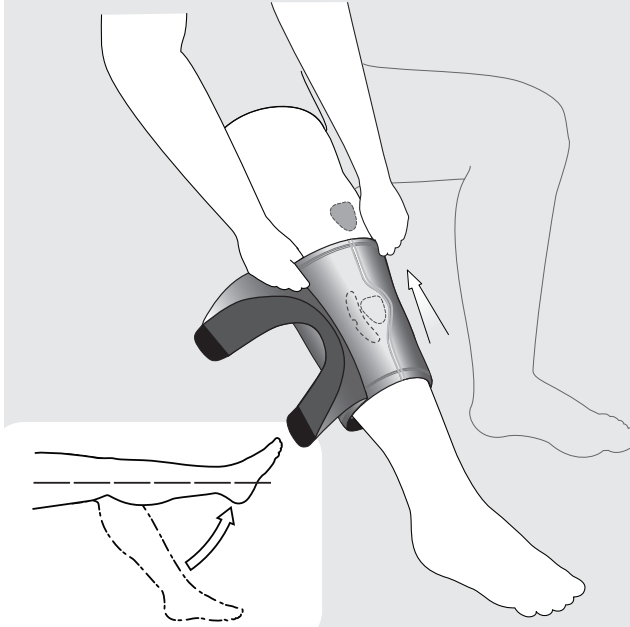
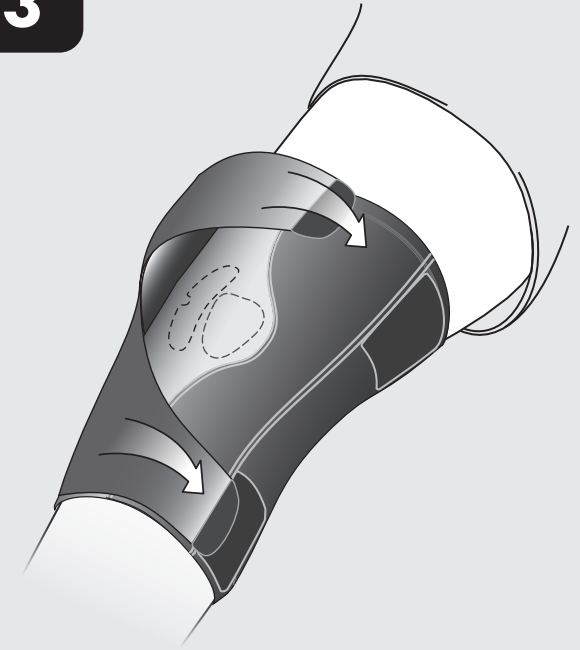
1**2****3****TIPS FOR USE AND CARE**

- Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.
- Do not machine wash or tumble dry.
- After exposure to salt water or dirt, thoroughly rinse the brace with fresh water and allow to air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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