**ENGLISH**  
**INTENDED USE/INDICATIONS:**  
 Knee Ligament bracing products are designed to provide stability and support following ACL, PCL, MCL and LCL injuries.

**INTENDED USERS:**  
 The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.




**CONTRAINDICATIONS:**  
 Not Applicable.

**⚠️ WARNINGS AND PRECAUTIONS:**

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.
- If you experience increased pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For sliding activities, it is recommended that a slide guard (pn.22000,22001) be worn with the brace. Wearing a slide guard is intended to minimize impact to the brace. The slide guard may not be effective in all situations.
- Care and cleaning of this product is essential to its continuing strength and performance. Follow standard and cleaning process described in instructions.
- When using braces bilaterally, modify activity as necessary to keep medial hinges from contacting each other.
- When using a Fusion brace equipped with the OA hinge it is important to keep dirt and grit out of the hinge mechanism to maintain proper hinge mobility and brace function.
- For single patient use only.

**INSTRUCTIONS FOR USE:**

- A.** Sit in a chair with your foot flat on the floor, and the knee bent at around 45 degrees. Unfasten all straps.  
 Slide the brace onto the leg positioning the center of the hinge with the top of your kneecap.
- B.** Press the shin cuff down onto the leg and fasten strap #1 behind your calf. Fasten strap #2 in the back above the knee.
- C.** Push the thigh cuff down against the leg and fasten strap #3. Fasten strap #4 in front below the knee. Fasten Strap #5 in the back below the knee.
- D.** Stand up and fasten strap #6 in front above the knee.
- E.** For PCL indications follow the strapping sequence in figure E and instructions above.
- F.** To remove the brace, simply loosen straps 4 & 6 (ACL version) or straps 5 & 2 (PCL version). Then, unfasten, and remove all the straps behind the knee and remove the brace. Fold the hook/loop tabs back against the straps. This will make it easier to re-apply the brace and allow the hinges to be placed in the proper posterior position.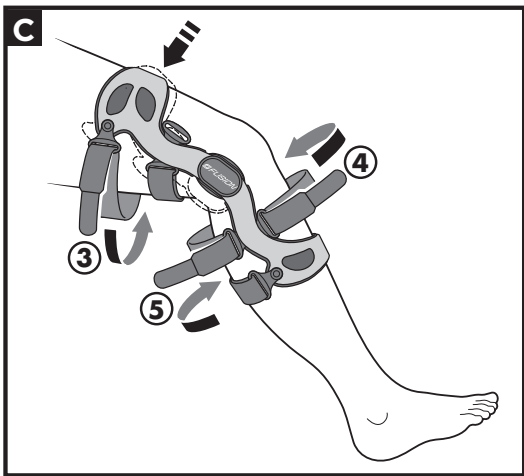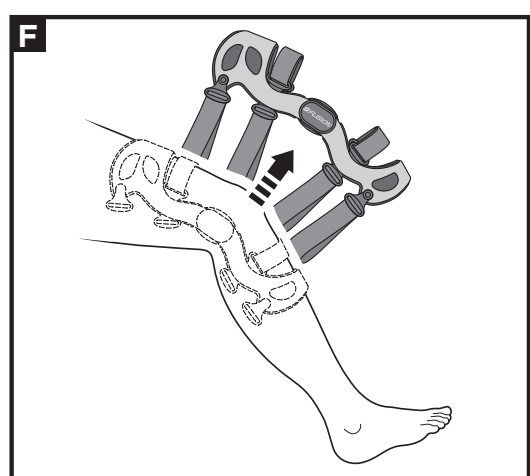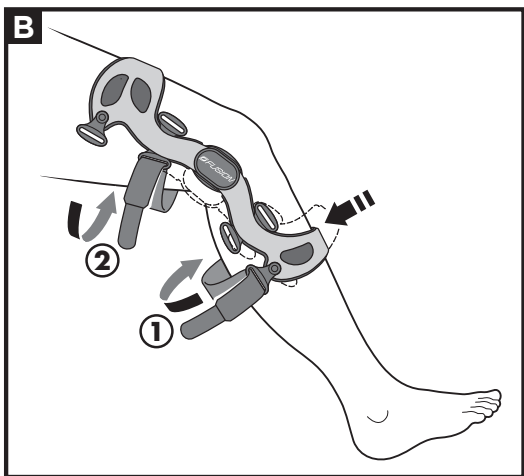

**HINGE RANGE OF MOTION ADJUSTMENT**  
 RÉGLAGE DE LA MOBILITÉ ARTICULAIRE  
 AJUSTE DEL RANGO DE MOVIMIENTO DE LA BISAGRA  
 REGOLAZIONE DELL'AMPIEZZA DI MOVIMENTO DELLO SNOODO  
 SCHARNIEREINSTELLUNG FÜR VERSCHIEDENE BEWEGUNGSGRAD E  
 JUSTIERUNG AV STÖTTENS BEVEGELSESRÅDE

To achieve the best possible fit of the brace straps, be sure that the strap pads on the inside of each strap are centered on each strap and provide a comfortable contact point between the skin and the strap.  
 Strap Length Adjustment: If a strap is too long or too short (preventing the numbered strap tab from properly adhering to the strap). First, remove the strap pad and unfasten the strap tab, then unfasten the "tail end" of the strap and slide the strap to either lengthen or shorten. Refasten the strap ends and replace the strap pads.

If there is no diagnosed or apparent instability associated with the unicompartmental knee OA, simplify the strap application for FUSION OA braces by fastening all straps behind the knee first (1,2,3, and 5) and then apply straps 4 and 6 in front of the knee.  
 Notice that the hinges may be positioned behind the bent knee, this is normal. As each strap is fastened, the hinges will move forward, until the brace is properly positioned on the leg.


**HINGE RANGE OF MOTION ADJUSTMENT**  
**(Figure 1)** Brace is shipped with a 10 degree extension stop installed. Additional flexion and extension stops are included.  
 To install a hinge stop:  
 • Remove the condyle pad  
 • Using a screwdriver remove the hinge screws  
 • Remove any stops already installed  
 • Insert required stops  
 • Replace screw and position the condyle pad in its correct position

**USE AND CARE:**  
 Straps must be removed and cleaned regularly depending upon your activity level and perspiration. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry. Occasionally apply a dry lubricant to the hinges especially after use in water.

**MATERIAL CONTENT:**  
 Aluminum, Ethylene-Vinyl Acetate (EVA), Nylon, Polyester, Polyurethane, Stainless Steel

**WARRANTY:**  
 Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

**DISPOSAL:**  
 Dispose of product according to local regulations.

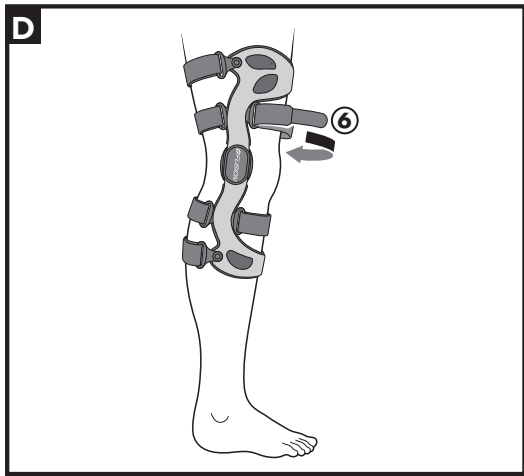**NOTICE TO USERS:** If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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