# FUSION<sup>®</sup> XT OA PLUS

## ADJUSTABLE HINGE INSTRUCTIONS

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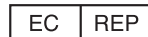
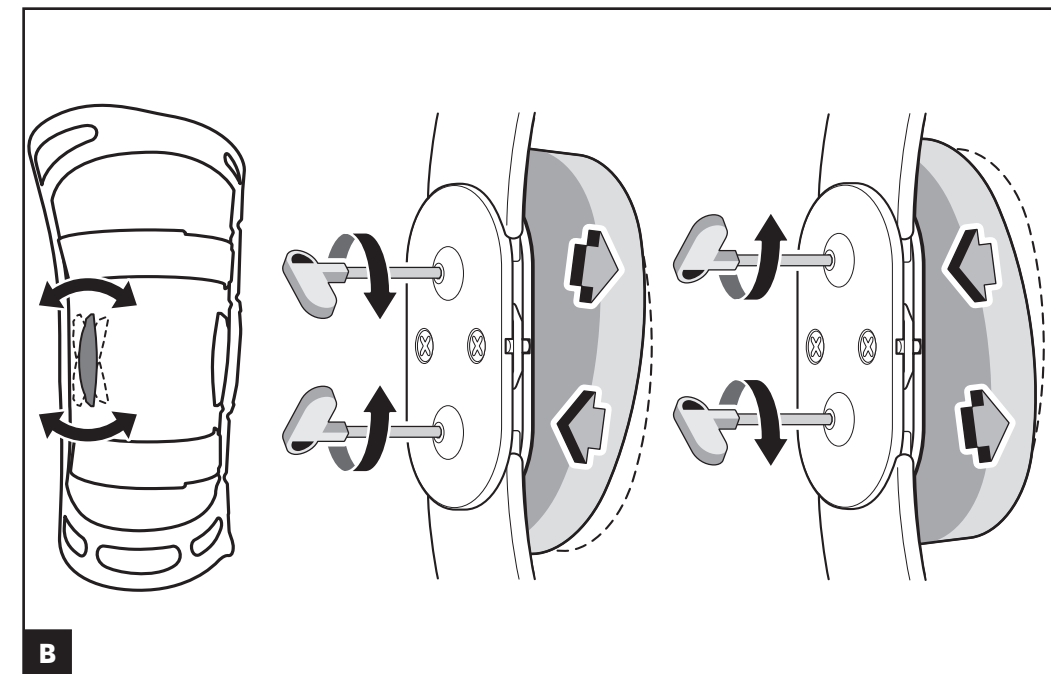
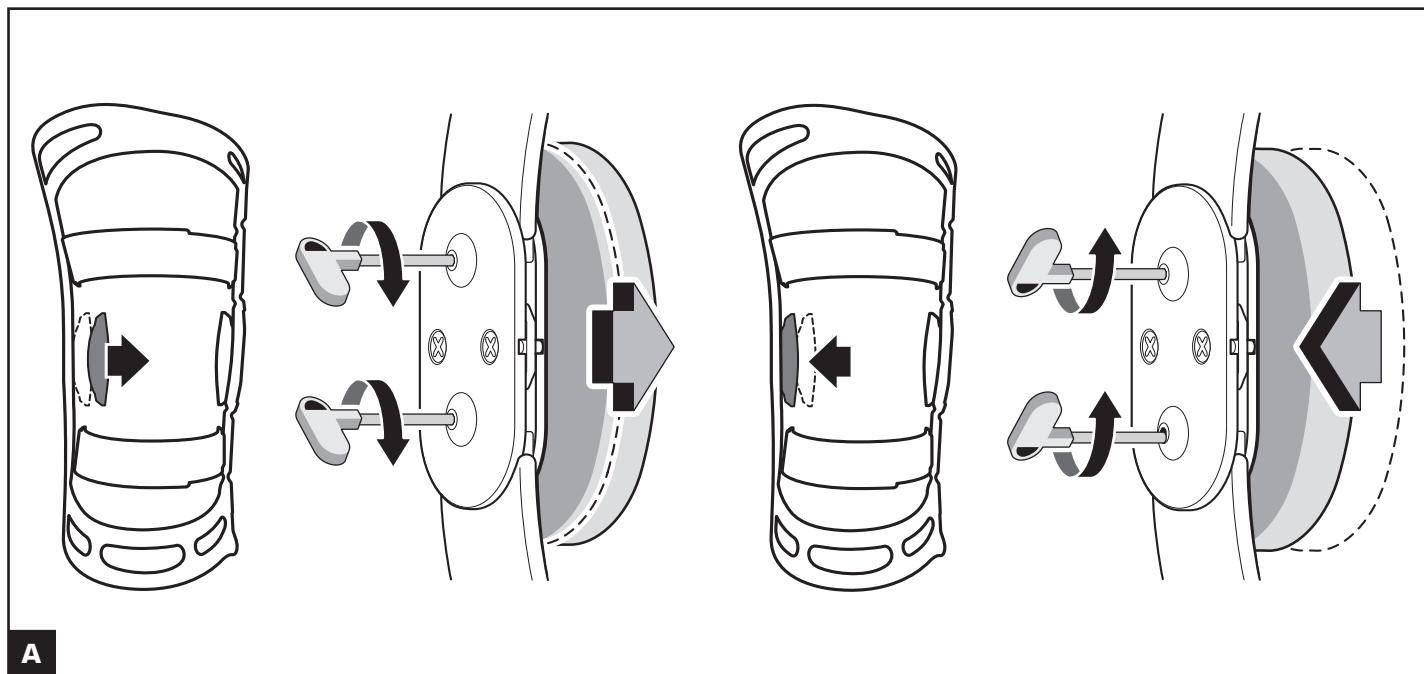
AW-1.00574 Rev C 07/23

## ⚠️ WARNINGS

**WARNING:** CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS. FAILURE TO PROPERLY POSITION THE BRACE AND FASTEN EACH STRAP WILL COMPROMISE PERFORMANCE AND COMFORT.

**WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

**WARNING:** FOR SLIDING ACTIVITIES, IT IS RECOMMENDED THAT A SLIDE GUARD (PN.22000,22001) BE WORN WITH THE BRACE. WEARING A SLIDE GUARD IS INTENDED TO MINIMIZE IMPACT TO THE BRACE. THE SLIDE GUARD MAY NOT BE EFFECTIVE IN ALL SITUATIONS.

**CAUTION:** CARE AND CLEANING OF THIS PRODUCT IS ESSENTIAL TO ITS CONTINUING STRENGTH AND PERFORMANCE. FOLLOW REMOVAL AND CLEANING PROCESS DESCRIBED IN INSTRUCTIONS.

**CAUTION:** WHEN USING BRACES BILATERALLY, MODIFY ACTIVITY AS NECESSARY TO KEEP MEDIAL HINGES FROM CONTACTING EACH OTHER.

**CAUTION:** WHEN USING A FUSION BRACE EQUIPPED WITH THE OA HINGE IT IS IMPORTANT TO KEEP DIRT AND GRIT OUT OF THE HINGE MECHANISM TO MAINTAIN PROPER HINGE MOBILITY AND BRACE FUNCTION.

**CAUTION:** FOR SINGLE PATIENT USE ONLY.

## Adjustable Hinge Instructions

### A ADJUSTMENT FOR LOOSE FITTING PADS

Use the tool provided to turn the upper and lower adjustment screws on the lateral hinge clockwise or counterclockwise. This will move the condyle pad inward, tighter against knee or outward (away from the knee). Adjust for a comfortable, consistent fit.

### B ADJUSTMENT IF TOO LOOSE OR TIGHT AT UPPER OR LOWER EDGE OF PADS

The condyle pad may be tilted to fit the contour of the knee. To tilt the condyle pad, adjust either the top or bottom screws. To do so, turn the screws counterclockwise to move the condyle pad edge outward, away from the knee. Turn the screws clockwise to move the condyle pad edge inward, tighter against knee.

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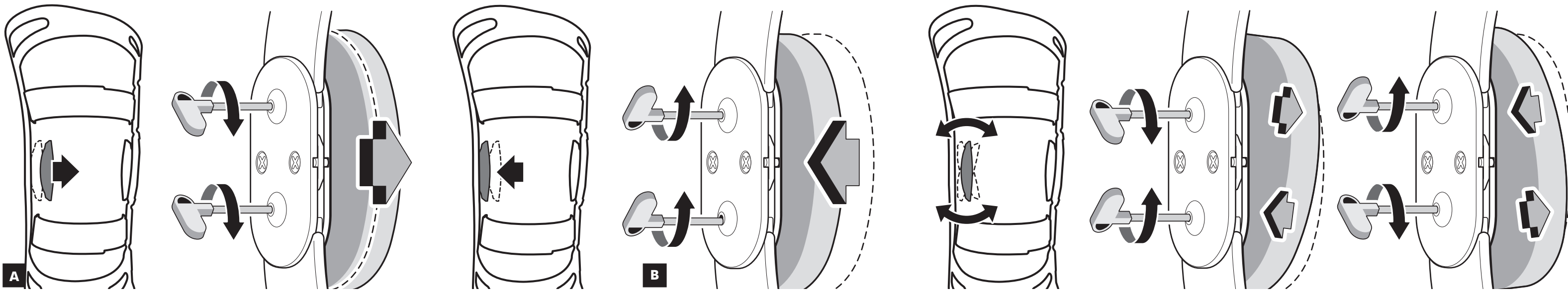
### Fusion XT OA Plus Unicompartmental Osteoarthritis Instructions

#### A ADJUST THE LATERAL (OUTSIDE) CONDYLE PAD

With brace on, walk 5 minutes to position brace properly and ensure that lateral condyle pad is applying lateral force to knee. If more lateral force is required to relieve symptoms turn upper and lower adjustment screws on side of hinge clockwise. This will move condyle pad inward (toward knee), applying more lateral force to knee.

- Upper and lower adjustment screws can be adjusted differently to accommodate various knee contours.
- For more comfort, different adjustments of upper and lower screws will distribute lateral force more evenly.
- Make sure large gel condyle pad is attached to the lateral (outside) condyle basket.

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### Fitting Instructions for loss or gain in leg girth:

#### B ADJUSTING THE LATERAL CONDYLE PAD INWARD OR OUTWARD

If there is a loss or gain in leg girth (swelling) the pressure against the knee from the condyle pads may be too great or insufficient. The lateral condyle pad may be adjusted to accommodate a change in leg girth.

##### 1 ADJUSTMENT FOR LOSS IN LEG GIRTH:

Use the tool provided to turn the upper and lower hinge adjustment screws clockwise. This will move the condyle pad inward, tighter against knee.

##### 2 ADJUSTMENT FOR GAIN IN LEG GIRTH (SWELLING):

Use the tool provided to turn the upper and lower hinge adjustment screws counterclockwise. This will move the condyle pad outward, away from knee.

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© 2023 Breg, Inc.