ENGLISH

INTENDED USE/INDICATIONS:

Provides fixed or range of motion control following injury or post-operative procedure to the elbow; stable fractures; biceps repair; and mild to severe soft tissue injuries.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

WARNINGS AND PRECAUTIONS:

- Carefully read firing instructions and warnings prior to use.
- If you experience increased pain, swelling, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

INSTRUCTIONS FOR USE:

- Loosen the bicep wrap (A), posterior bicep strap (B), posterior forearm strap (C), and the anterior bicep strap (D).
- Position the arm with approximately 20° flexion with thumb and anterior side of the elbow pointing upward as shown.
- Slide the brace onto the arm. Position the medial and lateral (inside and outside) side struts so that they are aligned with the centerline of the arm and the hinges are centered over the elbow joint.
- Fasten the bicep wrap (A).
- Fasten the posterior bicep (B) and forearm straps (C).
- Fasten the anterior bicep strap (D).
- Establish the desired angle of extension restriction.
- Adjust the length of the cross straps to hold the arm at the desired angle by sliding the straps through the plastic buckles, fasten the end tabs.

HINGE ADJUSTMENT INSTRUCTIONS

- Access the medial and lateral (inside and outside) hinges by pulling the top tabs on the neoprene hinge covers, pull the covers away from the hinge so that the hinges are fully accessible.
 - The brace is shipped with 10° hinge extension stops installed with additional flexion and extension stops in box. Stops are labeled in degrees:
 - Extension stops: 0°, 10°, 20°, 30°, 40°
 - Flexion stops: 45°, 60°, 75°, & 90°
 (brace is shipped with no flexion stops installed and their use is optional)
- To change the hinge stops, use a screwdriver to remove the screws. Remove existing stop, insert new stop, replace screws. Be sure that the screws are seated and snug. **There are hinges on both sides of the support. Be sure that both medial (inside) and lateral (outside) hinge stops are identical in degrees.**

USE AND CARE:

- Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry.
- Do not machine wash or tumble dry.
- After exposure to salt water or dirt, thoroughly rinse the brace with fresh water and allow to air dry.
- A dry lubricant, such as teflon spray, may be used on the hinge occasionally. Do not use wet lubricants.
- The hinge assemblies may be removed to allow easier washing of the sleeve and straps.
- Periodically inspect the sleeve, straps and hinges for excessive wear. Be sure all screws are tight and stops in place.

MATERIAL CONTENT:

Acetal, Aluminum, Cotton, Neoprene, Nylon, Stainless Steel

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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