1

Remove the liner from the boot by first unfastening all straps. Apply liner to the patient with the ankle at a 90-degree angle and the foot flat on the floor by wrapping the foot and ankle with the liner and fastening the hook & loop. **Note:** If additional padding is necessary to cover a sensitive area or for a more secure fit, you may use the provided spacer pads.

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2

Pull the uprights apart and slide the leg into the walker frame. Take care to position the ankle at a 90-degree angle and the uprights centered along the medial and lateral sides of the lower leg. Once the lower leg is in position, pull off the felt strip covering the inside of the uprights.

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3

Starting with either the proximal or distal strap, fasten all straps so that they are snug and the patient feels secure in the walker.

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4

After applying the walker, locate the blue pump bulb (IF AIR VERSION) and release valve on the top strap of the fracture walker liner.

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5

After locating the pump, squeeze repeatedly to fill the internal air bladder. The air bladder should be filled according to the patient's comfort level. If the liner becomes too full or if there is a need to deflate the liner, simply open the release valve lid and press the release valve. To re-inflate, repeat Step 4.

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## Integrity Walker Boot Instructions for Use

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## WARNINGS

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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Breg, Inc.