ENGLISH

INTENDED USE/INDICATIONS:

The Cold Therapy pad accessories are single patient use, non-sterile pads to provide localized cold therapy specifically for the body part to which it is applied.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Refer to Polar Care® device IFU.

⚠ WARNINGS AND PRECAUTIONS:

The Intelli-Flo Pad can be cold enough to seriously injure the skin. The Intelli-Flo Pad is non-sterile and for single patient use only. Do not attempt to sterilize the Pad. Follow these instructions and the Operating Instructions and Product Inserts provided with the Polar Care Kodiak Motorized cold therapy units (available at www.breg.com or 1-800-321-0607).

- **Discuss treatment with your licensed health care practitioner.** Provide a complete medical history including any reactions to cold. Certain medical conditions make cold-induced injury more likely. Ask your practitioner about potential adverse reactions and cold-induced injuries.
- **Use only according to your practitioner's instructions** regarding the frequency and duration of cold application and breaks, how and when to inspect the skin, and total length of treatment. Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
- **Apply an insulation barrier to gray plastic channeled side** (such as the Breg Polar Dressing provided with each 3x5 pad, Webril, Kerlix, cast padding, elastic bandage) that provides complete coverage between the Intelli-Flo Pad and skin. Do not let any part of the Intelli-Flo Pad touch skin. If a sterile dressing has been applied to the treatment site that does not completely cover the skin under the pad, use an additional insulation barrier. The Intelli-Flo Pad alone is too cold to be applied directly to the skin.
- **The Intelli-Flo 3x5 pad is only indicated for use on hand/wrist and foot/ankle applications. Application to any body part other than what is indicated could lead to cold-induced injury.**
- **Inspect the skin** under the Intelli-Flo Pad (by lifting the edge) as prescribed, typically every 1 to 2 hours. Do not use the Polar Care Kodiak if dressing, wrapping, bracing, or casting over the Intelli-Flo Pad prevents skin inspections. Stop using and contact your practitioner immediately if you experience any adverse reactions, such as: increased pain, burning, increased swelling, itching, blisters, increased redness, discoloration, welts, other changes in skin appearance, or any other reaction identified by your practitioner.

INSTRUCTIONS FOR USE:

1. **Apply the Intelli-Flo Pad** with the gray plastic channeled side towards the skin. Refer to the graphics. Do not wrap pad so tightly as to restrict normal circulation, as this can cause injury.
2. **Connect the Intelli-Flo Pad** couplings to the Polar Care Kodiak unit couplings. The Intelli-Flo Pad hose couplings are only compatible with the Polar Care Kodiak unit couplings.
3. **Follow the Polar Care Kodiak Operating Instructions and Product Insert** inside the pouch; both are affixed to the unit. Application Examples. Application types may vary.
4. **To end cold therapy**, disconnect electrical power and press the two metal tabs on the hose coupling and gently pull apart or remove the Intelli-Flo Pad from your body.

USE AND CARE:

- Hang the pad upside down to drain excess water into drain or sink.
- Hand wash the outside of the pad as needed with a soft cloth, mild soap and 30°C (86°F) water. Rinse thoroughly.
- Wipe dry before storing.
- DO NOT machine wash or tumble dry.

MATERIAL CONTENT:

Tricot/Ply Blend Foam, Dureflex Poly Film, EPDM rubber, Polypropylene

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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