Orthopedic Support

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SUPPORTS ORTHOPEDIQUES

SOPORTES ORTOPÉDICOS

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com

AW - 1.93031 Rev C 0113

CAUTION: FOR SINGLE PATIENT USE ONLY.
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

ORTHOPEDIC SUPPORTS

- Hand wash and thoroughly rinse support. Air dry. To prolong product life and reduce skin problems, wash regularly.
- Certain products have hinge bars that may be removed for cleaning. Normally, hinge lubrication is not required, however a dry lubricant, such as teflon spray may be used.
- After exposure to salt water, thoroughly rinse support in fresh water and air dry.
- Periodically inspect support for wear. Do not use if worn excessively.
- Do not wear support directly over open wounds.
- If you experience dermatitis or adverse skin reactions while using this product, consult your health care professional.

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