Mini-Walker (Aluminum Struts) Application Instructions

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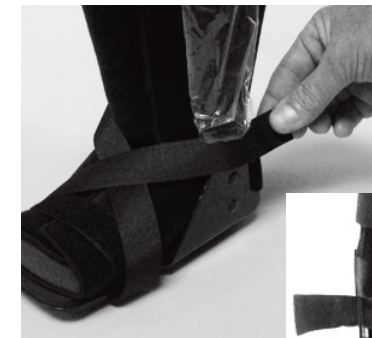
D

E

A. Parts included: Liner, walker, padding for proper fit, top strap, bottom strap, and small hook tabs.

B. Place the injured foot into the liner. Toes should push all the way through. Secure foot liner with hook and loop closure. Secure upper calf liner with hook and loop closure.

C. Unhook back of the figure eights strap from uprights. To loosen pull center of strap (do not remove strap all the way). Place foot into walker leaving upright plastic covers on. The uprights should be positioned along the side of the leg before removing covers. Secure to side of liners.

D. Pull figure eight strap to tighten criss-crossing straps over top of foot and wrap around and attach to upright. Slide back strap through uprights and secure hook and loop strap.

E. Wrap horizontal straps starting at the ankle and then near the top of the the walker. Small hook tabs may be used to help position strap to upright/liner if necessary.

Note: Hand wash the liner and straps with mild soap, air dry. Do not place liner or straps into a mechanical dryer.

Note: The Mini-Walker with Aluminum Struts should always be worn over a cotton sock. Consult your physician for further instructions on application and treatment.

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