ENGLISH INTENDED USE/INDICATIONS:

The intended use for wrist bracing products is to provide stability, support, and/or immobilization to the wrist joint for patients recovering from a wrist injury or surgery.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of brace, follow all instructions.
- If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

INSTRUCTIONS FOR USE:

APOLLO WRIST BRACE UNIVERSAL

- Open brace and position hand in wrist brace.
- The palmar stay should line up below the middle crease of the hand.
- Secure the thumb strap to the brace, aligning it between the thumb and index finger.
- Wrap the lacing straps around the wrist and attach to hook and loop. Readjust if necessary. The knuckles should be free for bending.

APOLLO WRIST BRACE WITH SPICA

Steps 1-3 follow instructions above.

- Wrap the wide thumb strap around the thumb and secure to hook.
- Secure the thinner strap between the thumb and index finger and attach to hook.
- Wrap the lacing straps around the wrist and attach hook and loop. Readjust if necessary. The knuckles should be free for bending.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Aluminum, Cotton, Nylon, Polyester

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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