Plantar Fasciitis

Night Splint

FITTING INSTRUCTIONS

ENGLISH.....	4
SPANISH.....	5
ITALIAN.....	6
GERMAN.....	7
SWEDISH.....	8
DANISH.....	9
FINNISH.....	10
ROMANIAN.....	11
POLISH.....	12
DUTCH.....	13
BAHASA MELAYU.....	14

Breg, Inc.

2382 Faraday Avenue, Suite 300
Carlsbad, CA 92008 U.S.A.

P: 800-321-0607


F: 800-329-2734

www.breg.com


©2024 Breg, Inc. All rights reserved.


AW-1.04427 Rev C 06/24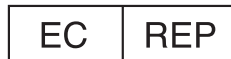



E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany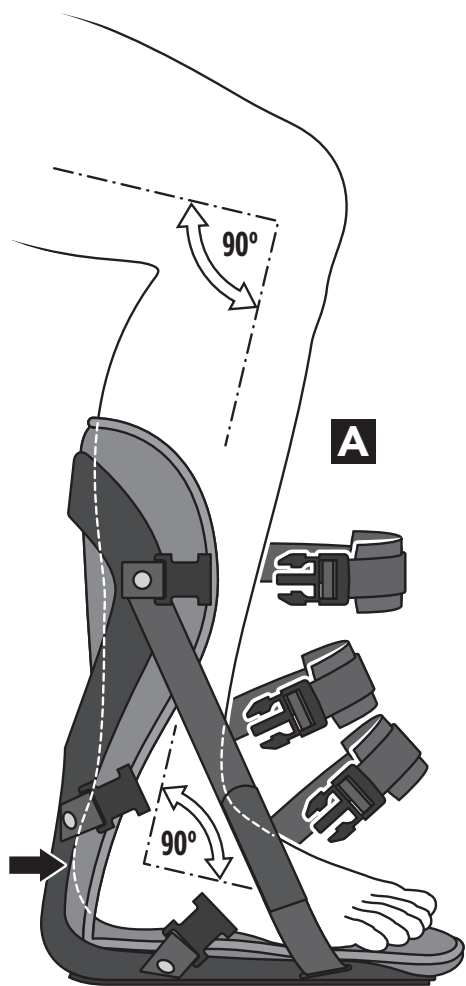

A

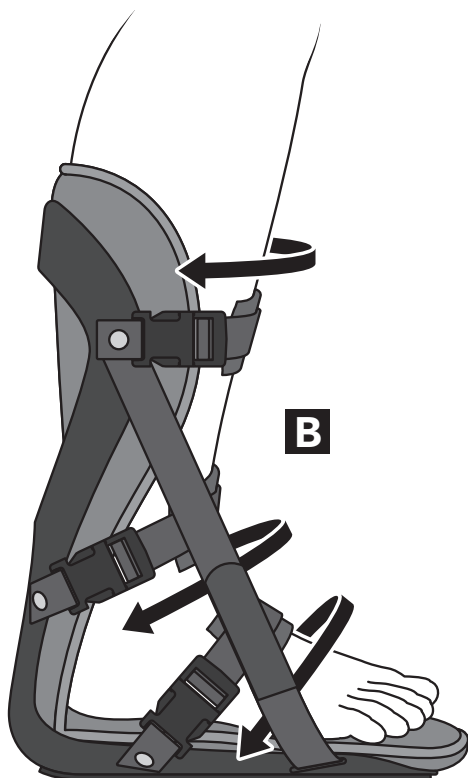B

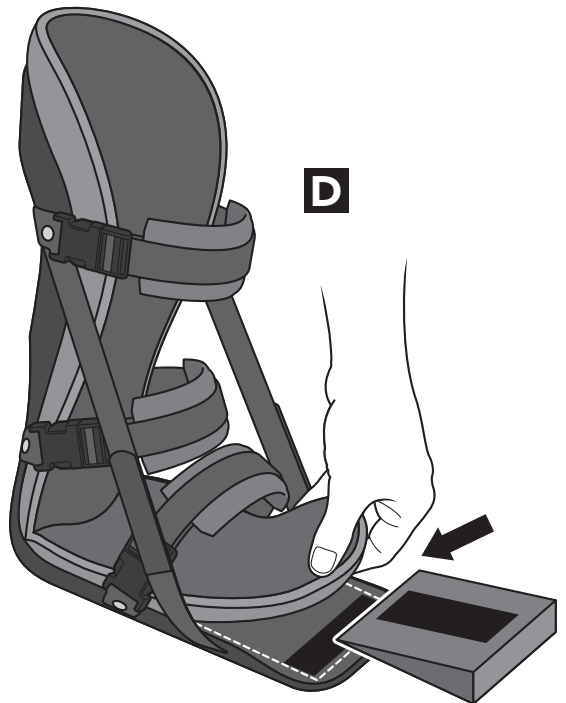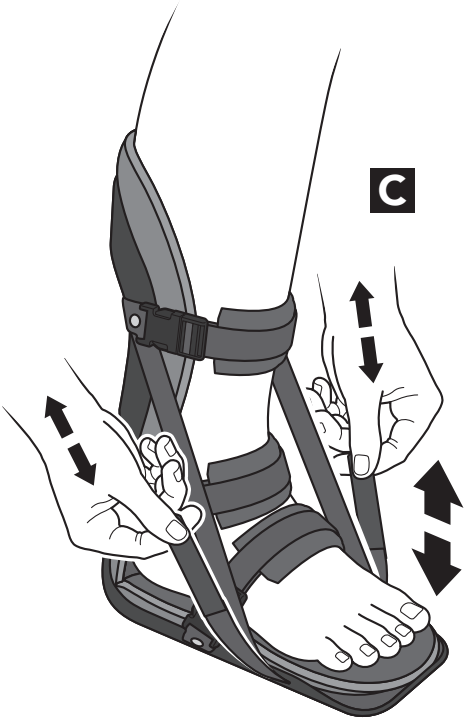⚠️ WARNINGS

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS.

WARNING: IF YOU EXPERIENCE OR HAVE AN INCREASE IN PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.


WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR MEDICAL PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: FOLLOW APPLICATION AND CARE PROCESS DESCRIBED IN INSTRUCTIONS.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CARE & WASHING INSTRUCTIONS:

REMOVE LINER FROM NIGHT SPLINT FOOT BASE. HAND WASH IN COLD WATER WITH A MILD DETERGENT, RINSE AND LAY FLAT TO AIR DRY.

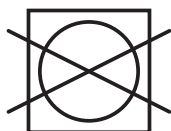A. Open all 3 buckles. Place the affected foot in the splint with knee flexed at 90°.

The heel should be at the very rear of the splint and the toes should not extend past the end of the splint.

The ankle should be at 90° when the splint is properly positioned.

B. Fasten the buckles. If necessary, adjust the buckles to ensure a secure and comfortable fit.

C. Release the hook and loop straps on each side of the foot bed. Pull up or lower the straps to adjust plantar fascia stretch.

D. To achieve additional stretch, pull the front portion of the liner away from the footplate and insert the 5° foam wedge between the liner and the footplate.

In the event of a serious accident in relation to the device, send a report to Breg and the competent authority of the Member State of residence of the user and / or patient.

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