ENGLISH

INDICATIONS FOR USE:

Breg's Polar Care® is intended for application of external cold therapy for temporary reduction of pain, swelling, and/or inflammation resulting from injury or surgery.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Non-sterile. Do not use directly over breached skin.
- For single patient use only.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
- Follow the licensing health care practitioner's protocol regarding method, frequency and duration of administration.
- Carefully read use instructions and warnings prior to operation.
- Carefully read use instructions and warnings provided with Polar Pad and associated Polar Care device.
- Do not attempt to sterilize this device by any means.

INSTRUCTIONS FOR USE:

1. Position pad on wrap with smooth side against the wrap.
2. Slip blue pad stem under short elastic band.
3. Slip upper portion of pad under long elastic band. Fold upper portion of multi-use pad into thirds first, then unfold.
4. Position the pad directly over the traumatized area of the body when the wrap is applied to the patient. Care should be taken to insure that the pad is not folded.
5. If the Breg Sterile Polar Dressing is to be used, apply it to the pad after the pad has been secured in the wrap.

USE AND CARE:

- Hang the pad upside down to drain excess water into drain or sink.
- Hand wash the outside of the pad as needed with a soft cloth, mild soap and 30°C (86°F) water. Rinse thoroughly.
- Wipe dry before storing.
- DO NOT machine wash or tumble dry.

MATERIAL CONTENT:

Nylon, Polyurethane

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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