**INDICATIONS:** Indicated for immediate application following injury to or reconstruction of the Achilles tendon.

**CONTRAINDICATIONS:** Contraindicated for unstable fractures or for fractures of the proximal tibia or fibula.

**WARNING:** Federal Law Restricts this Device to Sale by or on the Order of a Licensed Health Care Practitioner.

**WARNING:** For Single Patient Use Only.

**CAUTION:** Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

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**CAUTION:** This device will not prevent or reduce all injuries.

**CAUTION:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

**CAUTION:** This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

**CAUTION:** Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time.

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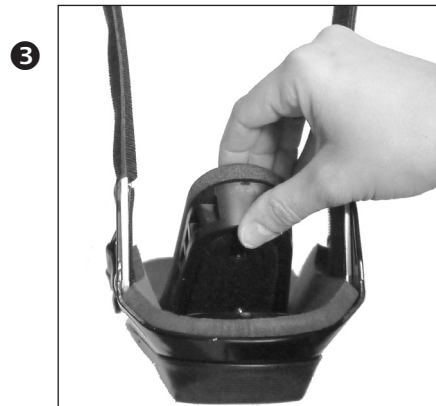
## ENGLISH

### 1 Remove Wedges

To change or remove the wedges, pry the upper wedge assembly up and save.

**NOTE:** When using the maximum number of wedges, ensure that the upper calf strap engages the uprights using at least half of the strap width.

### 2 Peel out each 10° wedge as necessary for patient progress.

**NOTE:** It is necessary to remove and refit the liner each time a wedge is added or removed. Straps should also be adjusted as to not interfere with the pump and release valve.

### 3 Replace the upper wedge assembly.

### 4 Replace Cutout

After all wedges are removed, the saved cut out piece can be replaced for further rehab.

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