ENGLISH

INDICATIONS FOR USE:

These braces are designed to provide immobilization and/or controlled range-of-motion for patients recovering from knee surgery or those that have knee injuries or instabilities.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- For Single Patient Use Only.
- Indicated for immediate application following injury or operative procedures to reduce pain and swelling and aid in healing.
- Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.
- This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.
- The range-of-motion stops on both the medial and lateral hinges must be set the same.
- This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

INSTRUCTIONS FOR USE:

- LOOSEN ALL FOUR STRAPS:** Pull each strap through the D-ring and attach it back on itself to prevent entanglement.
- FIT THE BRACE:** Hold the brace beside the leg to determine the proper length, aligning the hinge with the center of the knee.
- SET Calf AND THIGH LENGTH:** Lift the locking tab up to free the extension arm (at the desired arm - top or bottom). Slide the extension arms to desired length and release locking tab back in place to lock extension arm.
- APPLY BRACE TO LEG REVOLUTION 3**
 - With the leg extended, slide the brace underneath the leg with closure at front (Rear Closure brace = closures at back). Make sure the center of the hinge is aligned with the center of the knee. Align the arms of the brace along the midline of the leg. The hinges should be parallel with the knee joint.
 - Attach "1" gastroc strap (smallest strap located above the calf area). Wrap the strap around the leg and pull through the D-ring, then attach to double-sided hook, and trim off excess.
 - Attach "2" gastroc strap (largest strap located above the calf area). Wrap the strap around the leg and pull through the D-ring, then attach to double-sided hook, and trim off excess.
 - Attach "3" gastroc strap (smallest strap located above the calf area). Wrap the strap around the leg and pull through the D-ring, then attach to double-sided hook, and trim off excess.
 - Attach "4" gastroc strap (largest strap located above the calf area). Wrap the strap around the leg and pull through the D-ring, then attach to double-sided hook, and trim off excess.
 - Attach "5" gastroc strap (smallest strap located above the calf area). Wrap the strap around the leg and pull through the D-ring, then attach to double-sided hook, and trim off excess.
 - Attach "6" gastroc strap (largest strap located above the calf area). Wrap the strap around the leg and pull through the D-ring, then attach to double-sided hook, and trim off excess.
 - Attach "7" gastroc strap (smallest strap located above the calf area). Wrap the strap around the leg and pull through the D-ring, then attach to double-sided hook, and trim off excess.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Aluminum, Cotton, Nylon, Polyester, Polyurethane, Stainless Steel.

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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