Shell Air Ankle Walker Application Instructions

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AW-1.00273 Rev C 0512

The decision as to when apply this brace is at the discretion of the physician or healthcare professional. Please read these instructions completely to help provide proper fit.

A. Choose the correct walker size for the patient. The end of the patient's toes should not extend past the toe of the walker.

B. Remove the anterior (front) plastic panel from the walker. Open the hook and loop closures on the walker liner in preparation for placing the patient's foot in the liner. Place the sock on the patient's foot.

C. Position the patient's foot in the walker liner, sliding the patient's heel in proper position as far towards the heel area of the walker as possible.

D. Secure the liner around the patient's foot and lower leg with the hook and loop closures. Attach the toe cover by folding it over the toes and securing the hook and liner. Note: The toe cover can be removed if desired.

E. Position the anterior plastic panel over the walker liner with the hook tabs on the panel holding it in place.

F. Beginning with the toe strap, feed the straps through the loops and secure the hook and loop closures. The straps should then be adjusted to ensure a secure and comfortable fit. Caution should be taken to not over tighten the straps.

G. The liner has dual air chambers that are inflated separately.

H. The inflation tubes are located on the sides of the walker and can be re-positioned on the liner if desired.

I. Inflate the air liner chambers by placing the CLEAR end of the inflation bulb into the air tubes and pump with one hand. Both air chambers should be inflated to provide even compression.

J. Caution: DO NOT OVER INFLATE THE AIR CHAMBERS.

K. To release air from the chambers, place the BLACK end of the inflation bulb into the air tubes and pump to deflate.

L. The air chambers and inflation tubes can be removed from the liner if desired.

Note: This product is designed for single patient use. The walker should be worn with a sock. To prevent irritation or rash, the liner or sock can be hand washed in mild soap and rinsed thoroughly. AIR DRY ONLY. Additional pads are included if needed.

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