## ENGLISH

### INTENDED USE/INDICATIONS:

Indicated for post rotator cuff procedures, labral repair, lesions, dislocations, joint reconstruction.

### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Not Applicable.

### ⚠️ WARNINGS AND PRECAUTIONS:

- For single patient use only.
- If you experience increased pain, swelling, or any adverse reactions while using this product, immediately consult your medical professional.
- Carefully read fitting instructions and warnings prior to use.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program.

### INSTRUCTIONS FOR USE:

- Place injured arm in sling.
  - Put opposite arm through harness
  - Attach harness to buckle on the top of the sling.
- Make adjustments to fit patient.
  - Tighten posterior shoulder straps as necessary
- Attach the closure strap at the elbow and thumb, to hold the arm in place.
- Adjust the elbow relief door to tighten the sling or relieve olecranon pressure
- To remove the sling, unbuckle the straps and loosen the closure straps then slide the arm out of the sling.
- Applying the abduction pillow  
*Abduction pillow configuration should be determined by clinician.*
- Abduction pillow configurations
  - 45° Neutral
  - 90° Neutral
  - 15° Abduction
- Applying the exercise ball

### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

### MATERIAL CONTENT:

Acetal, Cotton, Nylon, Polyester, Polyurethane.

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see [www.breg.com](http://www.breg.com)

### DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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