
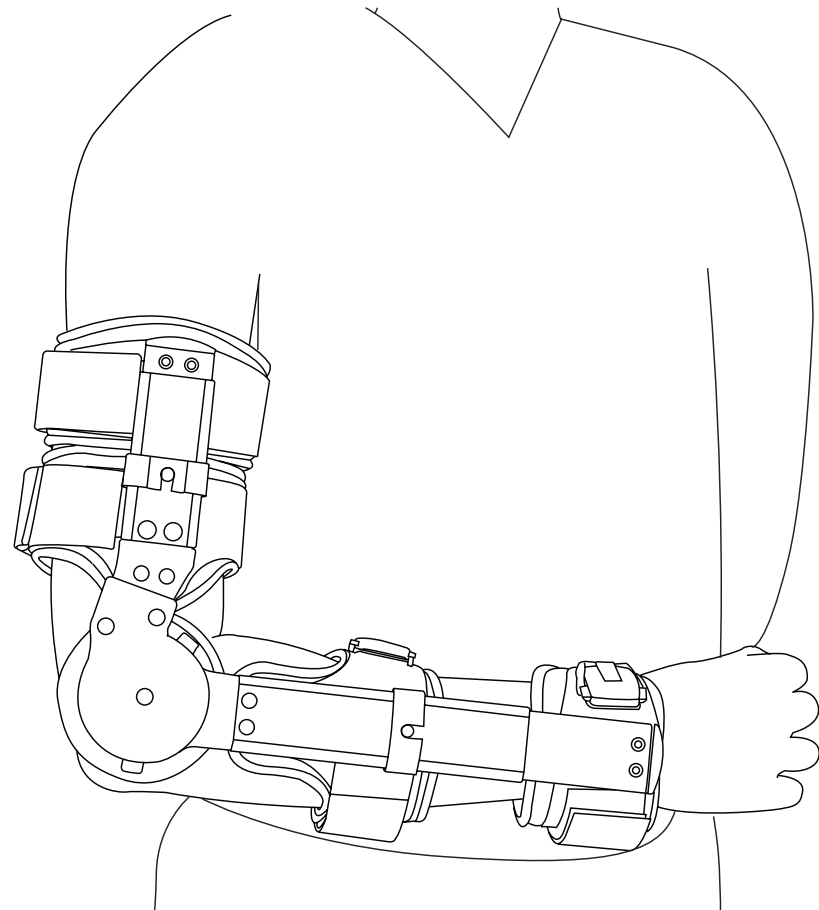
T-Chek Elbow Brace

Instructions for Use CP020244 Rev D 09/15

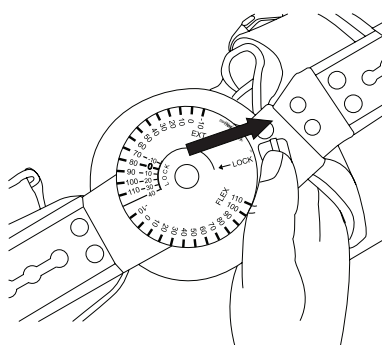1

1. Unlock the Hinge
The Telescoping Elbow Brace is packaged in the locked position. Slide the orange button away from the hinge to unlock the brace.

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2a

2. Position the Brace
a. There are four over-center buckles on the Elbow Brace. Pinch the two gray tabs on the buckles to open.
b. Position the hinge over the lateral humeral epicondyle (outer elbow).

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
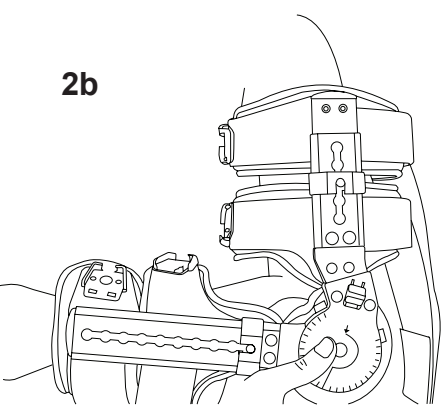
2b

3a

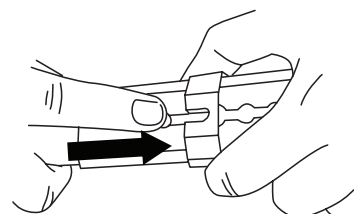3. Adjust the Arm Lengths
a. While the hinge is aligned with the elbow (see Step 2b), make length adjustments to the forearm. Slide the locking clip over to expose the button, depress the button and extend the arm to the desired length (wrist bone).
b. Secure the locking clip by sliding it over the button to lock the button in place.
c. Repeat the same process to adjust the humeral arm length.

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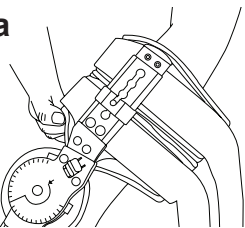
4a


4. Apply the Brace
a. Slide the arm into the brace.
b. Form the black aluminum plates around the forearm.
c. Form the black aluminum plates around the upper arm.

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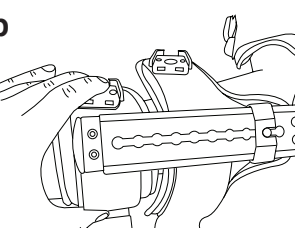
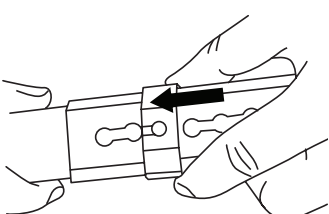
3b


5a

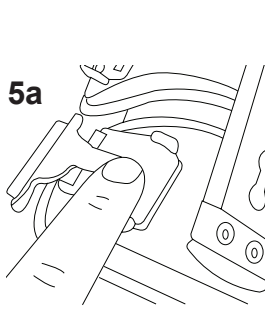
5. Secure the Straps
a. Connect the over-center buckle located closest to the wrist.
b. Adjust the strap tension using the hook to pile fastener at the strap end.
c. If necessary, trim excess strap by removing the hook piece, trimming the strap and replacing.
d. Repeat this process for the remaining three straps.

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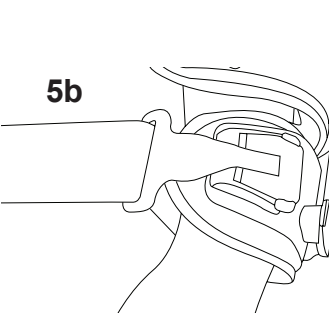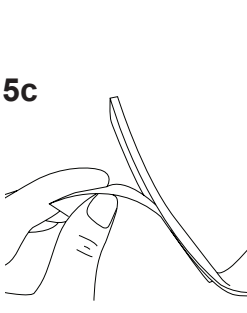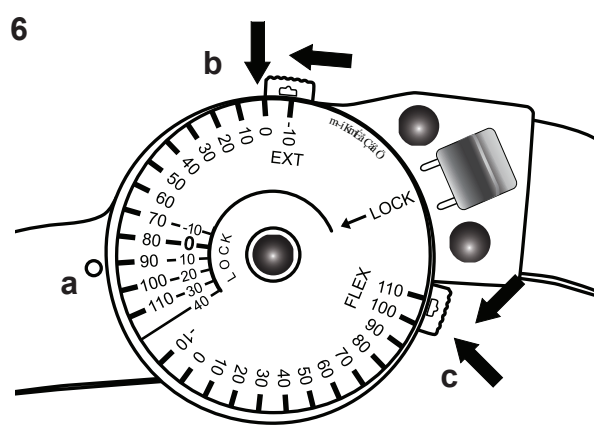
6. Adjusting the Hinge
a. Move arms to the desired position between the prescribed range of motion (ROM).
b. Press the gray extension button and slide it to the desired setting. The button will spring out to set the extension range of motion.
c. Press the gray flexion button and slide it to the desired setting. The button will spring out to set the flexion range of motion.

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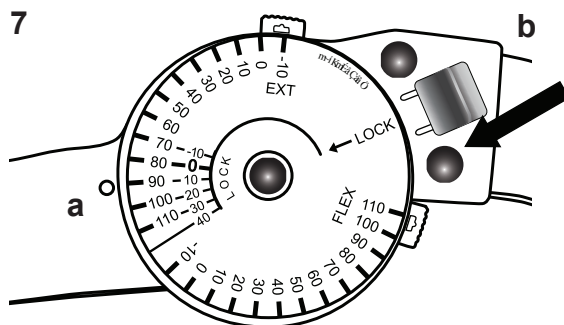
7. Locking the Brace (optional)
a. The Elbow Brace can be locked at a prescribed setting between -10° and 100° in 10° increments. To lock the brace, move the uprights aligning the indicator mark to the prescribed setting in orange.
b. Slide the orange lock tab toward the hinge. This will lock the brace in place.

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
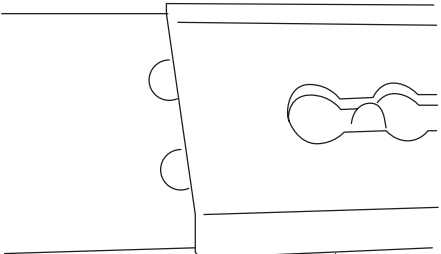
Hand Position Attachments (Optional)


1a

1b

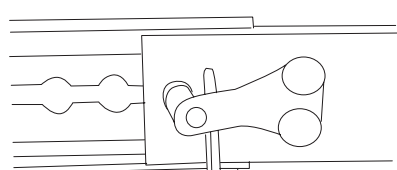1c

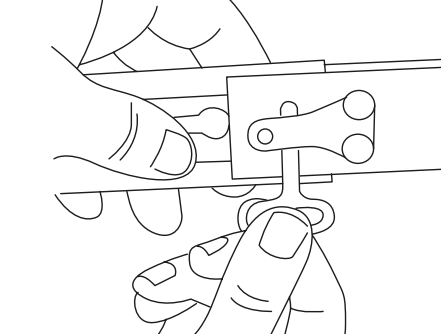1. Attach the Hand Attachment

- Insert key (provided) behind spring by depressing the button and sliding the key between the spring and upright.
- Turn the key 90°. Slide out the forearm extension.
- Slide the Hand Attachment into the brace and stop at desired length. Turn the key 90° and remove.

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2

2. Form the Hand Plate (Neutral Hand Position Attachment Shown)

With the wrist in the desired position, form the hand plate around the wrist and hand.

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This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

Indications: Indicated for immediate application following injury or operative procedures to the elbow, or for stable or internally fixed fractures of the distal humerus or proximal to middle radius or ulna where locked or limited motion control of the elbow is desired.

Contraindications: Contraindicated for unstable fractures or for fractures of the proximal humerus, distal radius or ulna.

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For product information or questions pertaining to sales or service, please contact the national distributor in your area or Breg directly.

© 2015 Breg, Inc. All Rights Reserved
2885 Loker Ave E
Carlsbad, CA 92010 USA
Toll Free Tel: 1-800-321-0607
Local: +1-760-795-5440
Fax: +1-760-795-5295
www.Breg.com

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.**

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