**ENGLISH**

**INTENDED USE/INDICATIONS:**

Provides fixed or range of motion control following injury or post-operative procedure to the elbow; stable fractures; biceps repair; and mild to severe soft tissue injuries.

**INTENDED USERS:**

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**CONTRAINDICATIONS:**

Not Applicable.

**⚠️ WARNINGS AND PRECAUTIONS:**

- Carefully read use/care instructions and warnings prior to use.
- Do not remove T Scope elbow brace unless instructed by your medical treatment professional. Do not change range of motion hinge settings without supervision by a medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your medical treatment professional regarding safe and appropriate activity level while wearing this device.
- If you experience increased pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
- For single patient use only.

**INSTRUCTIONS FOR USE:**

**FITTING THE T SCOPE ELBOW PREMIER:**

- Unlock the telescope locks (A) on the forearm and humeral bars by rotating as shown. Hold the brace up to the arm (example shows right arm) with the hinge centered over the elbow joint. The hinge lock (B) on the hinge must be facing the patient's hand. Slide telescoping bars until they match the length of the upper arm and forearm, lock the telescope locks (C) by rotating as shown.
- While supporting the injured arm, fasten the forearm strap closest to the hinge (i), then the upper arm strap closest to the hinge (ii).
- Next, fasten the upper arm strap furthest from the hinge (iii), lastly, fasten the strap closest to the hand (iv).

**CONTROLLING RANGE OF MOTION:**

The spring loaded tabs may be adjusted to limit range of motion.

(A) Extension settings between 10° hyperextension and 120° and (B) flexion settings between 10° and 120° may be selected by pulling the spring loaded tab out and sliding it to the desired position.

(C) To prevent range of motion, lock the T Scope Elbow Premier brace by sliding hinge lock toward the center of the hinge after positioning the arm at the appropriate angle. For additional security, position the spring loaded tabs at the appropriate lock-out angle.

**APPLYING THE SHOULDER STRAP:**

- Position the shoulder strap over the patient's shoulder and head with the strap pad located behind the neck. The D-rings on each end of the strap will attach to the upper arm strap closest to the hinge (v) and the forearm strap closest to the hand (vi).
- Attach the upper arm strap by sliding the alligator tab at the end of the strap through the D-ring on the end of the shoulder strap. Pull the strap completely through the D-ring and fasten firmly.
- Next, attach the forearm strap (vii) by sliding the alligator tab at the end of the strap through the D-ring on the end of the shoulder strap. Pull the strap completely through the D-ring and fasten firmly.

**USE AND CARE:**

Straps and foam pads may be removed by sliding the straps out of the strut housings. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly, air dry. Do not place the straps or foam pads into a mechanical dryer.

**MATERIAL CONTENT:**

Aluminum, Nylon, Polyacetal, Polyurethane, Stainless Steel.

**WARRANTY:**

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

**DISPOSAL:**

Dispose of product according to local regulations.

**NOTICE TO USERS:**

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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