
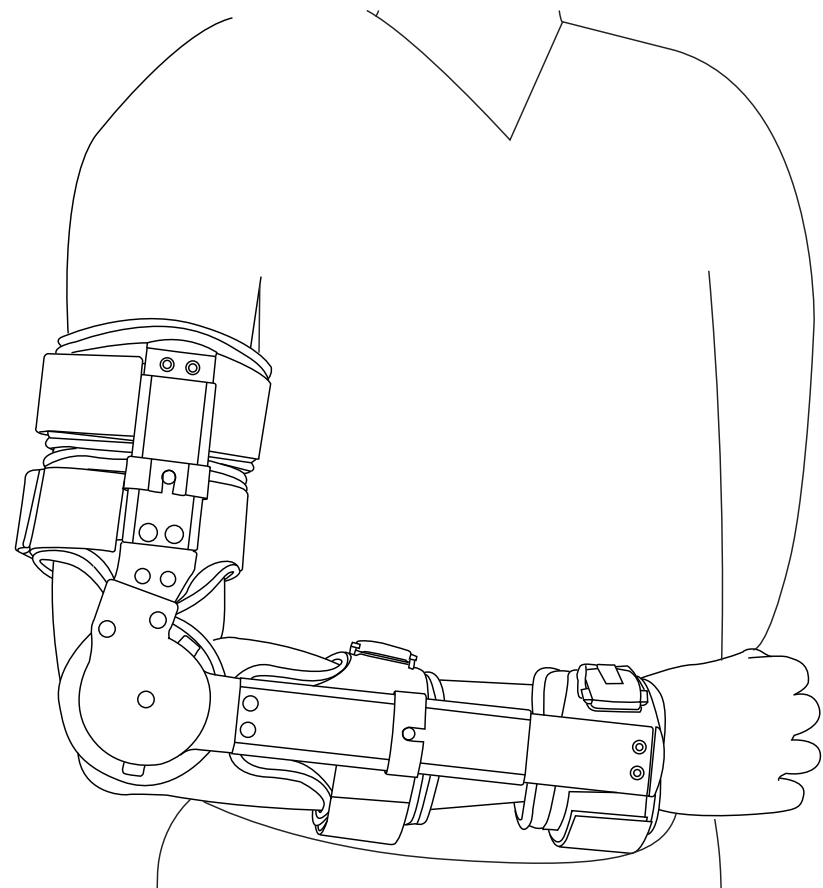
Telescoping Elbow Brace

INSTRUCTIONS FOR USE AW-CP020189 Rev C 1015

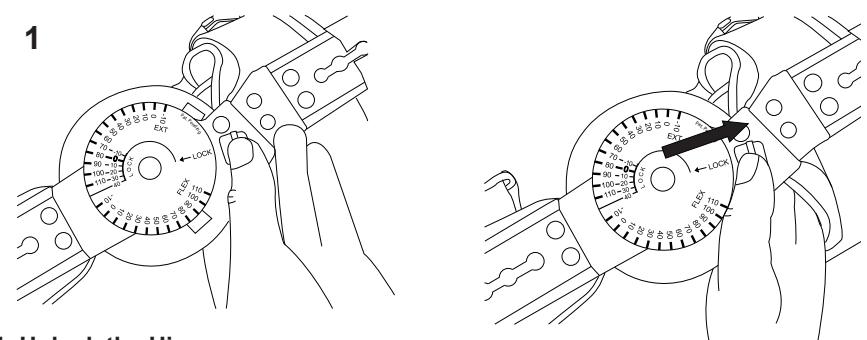ALTERNATIVE TO CASTING OR SPLINTING WITH RANGE OF MOTION

1. Unlock the Hinge

The Telescoping Elbow Brace is packaged in the locked position. Slide the orange button away from the hinge to unlock the brace.

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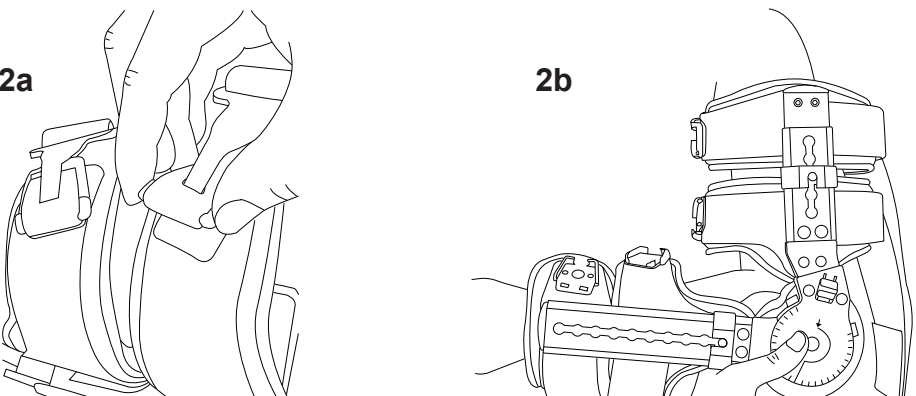
2. Position the Brace

a. There are four over-center buckles on the Elbow Brace. Pinch the two gray tabs on the buckles to open.
b. Position the hinge over the lateral humeral epicondyle (outer elbow).

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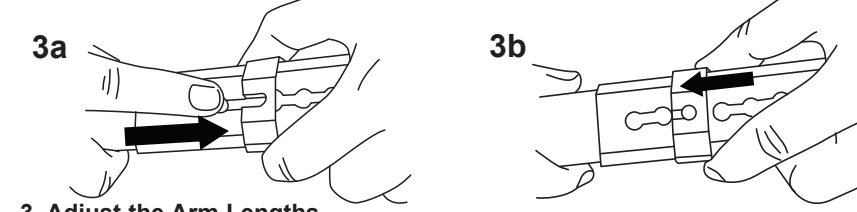
3. Adjust the Arm Lengths

a. While the hinge is aligned with the elbow (see Step 2b), make length adjustments to the forearm. Slide the locking clip over to expose the button, depress the button and extend the arm to the desired length (wrist bone).
b. Secure the locking clip by sliding it over the button to lock the button in place.
c. Repeat the same process to adjust the humeral arm length.

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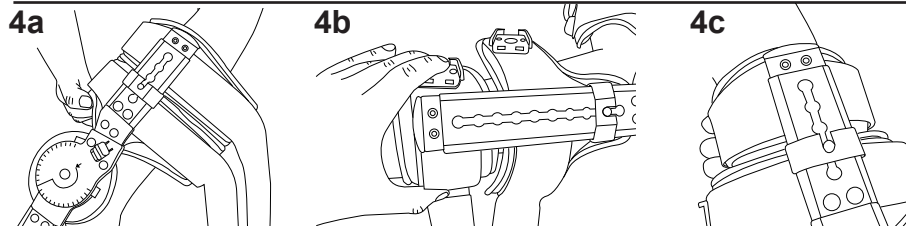
4. Apply the Brace

a. Slide the arm into the brace.
b. Form the black aluminum plates around the forearm.
c. Form the black aluminum plates around the upper arm.

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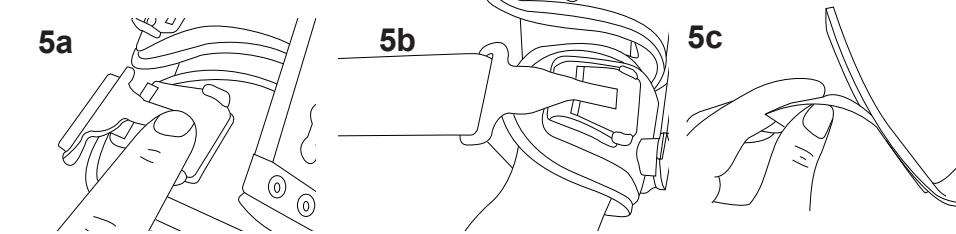
5. Secure the Straps

a. Connect the over-center buckle located closest to the wrist.
b. Adjust the strap tension using the hook to pile fastener at the strap end.
c. If necessary, trim excess strap by removing the hook piece, trimming the strap and replacing.
d. Repeat this process for the remaining three straps.

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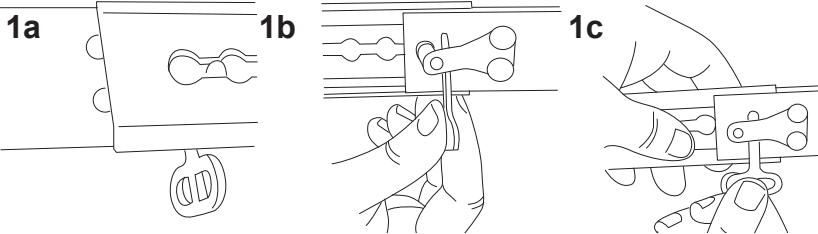
Hand Position Attachments (Optional)

1. Attach the Hand Attachment

a. Insert key (provided) behind spring by depressing the button and sliding the key between the spring and upright.
b. Turn the key 90°. Slide out the forearm extension.
c. Slide the Hand Attachment into the brace and stop at desired length. Turn the key 90° and remove.

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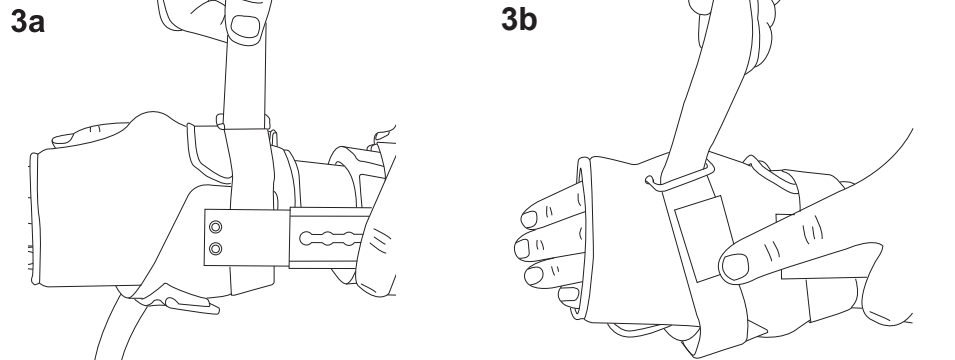
2. Form the Hand Plate (Neutral Hand Position Attachment Shown)

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3. Secure the Hand (Neutral Hand Position Attachment Shown)

a. Fasten the wrist strap in the same manner as the other brace straps.
b. Wrap the hand strap around the hand at the palm. Engage the hook to pile fastener on the hand plate. *NOTE: Do not tension wrist strap too tight as irritation may occur at the web of the thumb.*

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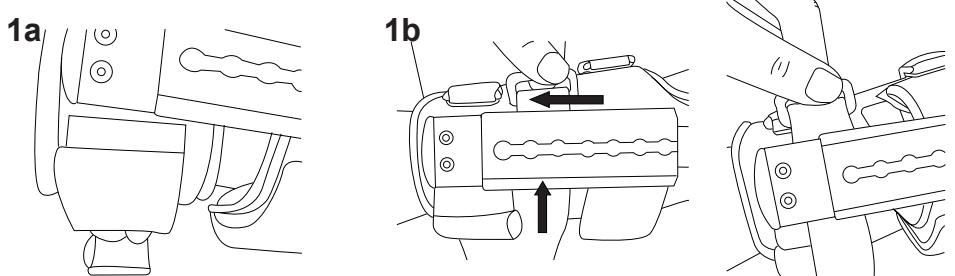
Sling Application (Optional)

1. Attach the Sling Strap

a. Attach the hook fastener to the strap under the wrist.
b. Thread the D-ring end of the sling strap between the black aluminum plates on the forearm. Press and engage the hook to pile fastener.

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2. Wrap the Sling Strap

Wrap the sling strap under the wrist, around the side of the waist on the unaffected arm, and up and over the shoulder of the affected arm.

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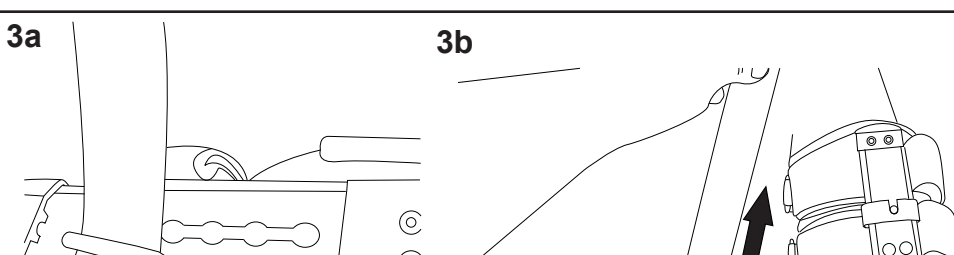
3. Adjust the Strap

a. Loop the free end of the strap through the D-ring.
b. Pull back to tension and engage hook to pile fastener.

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This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

Indications: Indicated for immediate application following injury or operative procedures to the elbow, or for stable or internally fixed fractures of the distal humerus or proximal to middle radius or ulna where locked or limited motion control of the elbow is desired.

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Contraindications: Contraindicated for unstable fractures or for fractures of the proximal humerus, distal radius or ulna.

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.**

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