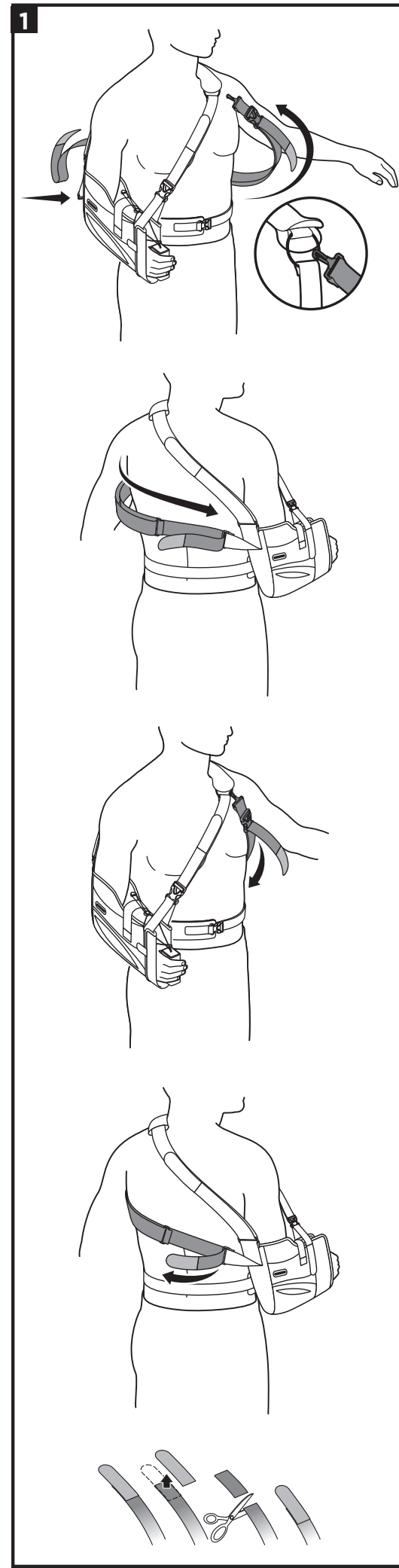
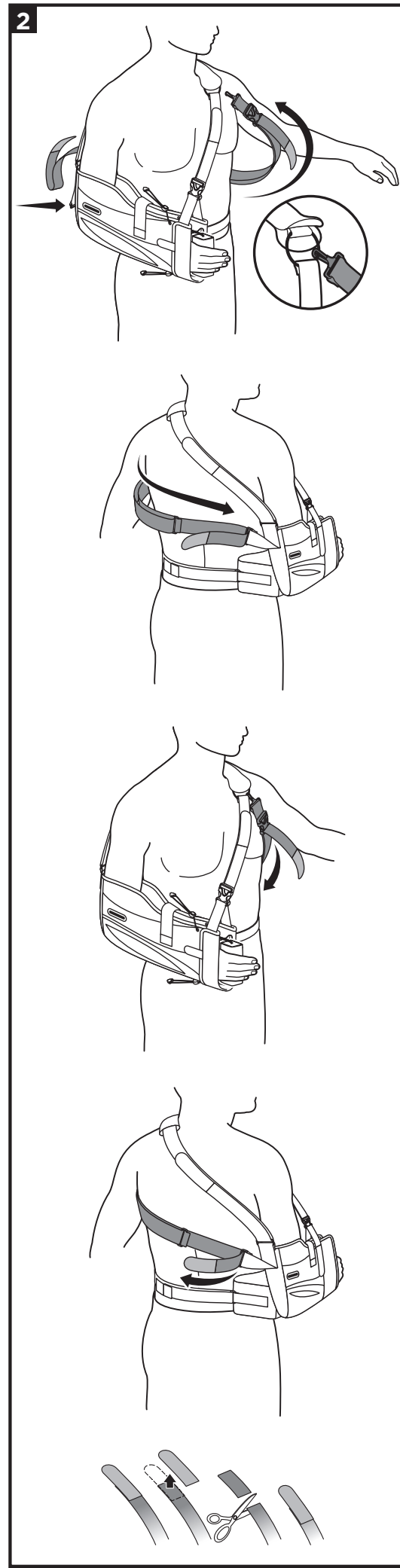
## ARC 2.0

## ARC 2.0 WITH PILLOW

## ENGLISH

### INDICATIONS FOR USE:

Shoulder bracing products is to provide alignment, support, and/or immobilization to the shoulder joint for patients recovering from shoulder injury or surgery.

### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Not applicable.

### ⚠️ WARNINGS AND PRECAUTIONS:

- For single patient use only.
- This device is not intended for extended use beyond 180 days.
- Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.
- This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.
- This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.
- Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65°C.) for any extended period of time.

### INSTRUCTIONS FOR USE:

#### APPLYING THE UNDERARM STRAP TO THE ARC 2.0 (1.) AND TO THE ARC 2.0 WITH PILLOW (2.)

Example: Right Shoulder

Attach clip to the ring inside neck pad and pass underarm strap under the unaffected shoulder. Attach loose end of the underarm strap to the D-ring at the back end of the sling. Loosen the hook closures at either end of the strap and pull the strap ends, front and back, to a comfortable tension. Refasten the closures.

**NOTE:** Excess strap may be trimmed if necessary.

### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Air Dry. Do not machine wash or tumble dry. If not rinsed thoroughly, residual soap can cause skin irritation or deterioration of the material.

### MATERIAL CONTENT:

Nylon, Polyester, Polyurethane

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see [www.breg.com](http://www.breg.com)

### DISPOSAL:

Dispose of product according to local regulations.

**NOTICE TO USERS:** If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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