# Vectra® Lite Fixed Walker Vectra Basic Fixed Walker Vectra Premium Fixed Walker

**Breg, Inc.**  
2382 Faraday Avenue, Suite 300  
Carlsbad, CA 92008 U.S.A.  
P: 800-321-0607  
F: 800-329-2734  
AW-1.90271 REV D 1123

EC REP

E/U authorized representative  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

**W A R N I N G**

**WARNING:** CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.  
**WARNING:** DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.  
**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.  
**WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.  
**WARNING:** TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.  
**WARNING:** DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.  
**CAUTION:** FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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## Vectra Lite, Basic and Premium Fixed Walker

- A Apply liner to leg.** With patient seated, leg(s) bent and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- B Slide walker onto leg over the liner straight up directly under foot.**
- C Fasten straps.** Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length).
- D Position walker with uprights centered over the malleoli (ankle bones).** Ensure heel is positioned at the rear of the Walker.
- E Remove quick-pulls from the uprights and discard.** Attach uprights to the liner. Press firmly.
- F Fasten straps.** Affix straps 3 through 5 to the outside of the uprights by alternating the buckle placement. Then fasten straps 3 through 5.

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
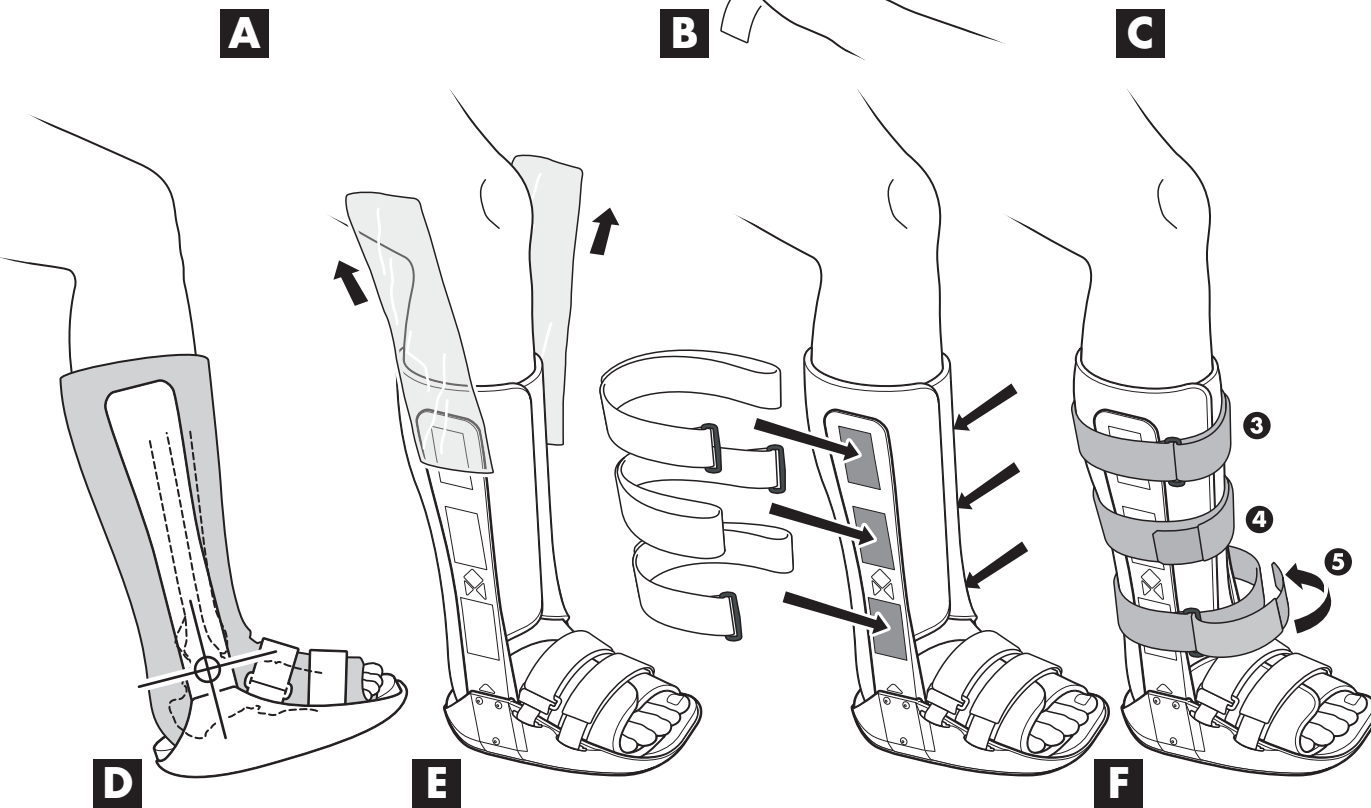
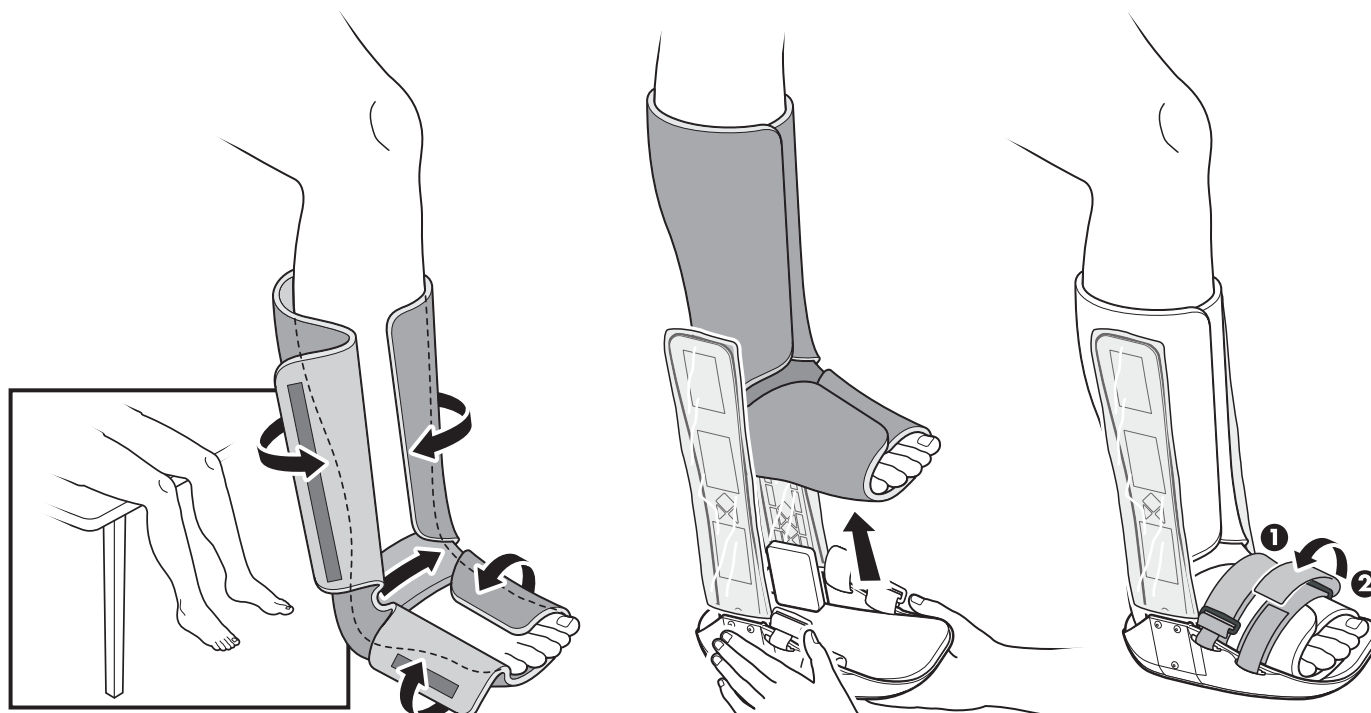
**Optional:** For maximum liner fixation and patient comfort, attach the 1 x 1/2" hook pieces to the liner at malleoli foam pads (or on the foot bed of the walker).

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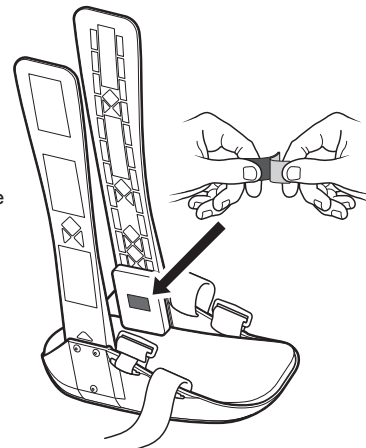
## Vectra Premium Fixed Walker

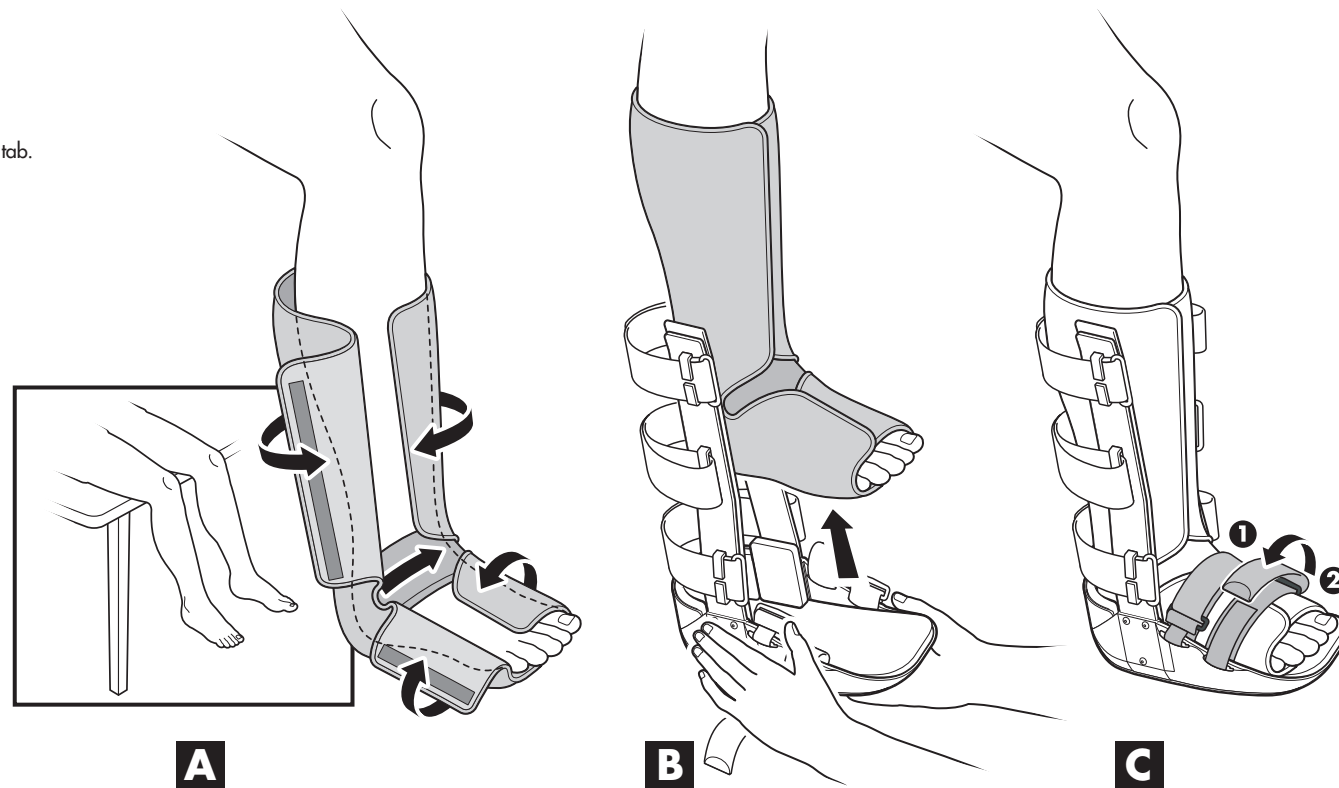- A Apply liner to leg.** With patient seated, leg(s) bent and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- B Slide walker onto leg over the liner straight up directly under foot.**
- C Fasten straps.** Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length by removing tab at end of straps, trimming and reattaching tab).
- D Position walker with uprights centered over the malleoli (ankle bones).** Ensure heel is positioned at the rear of the Walker.
- E Remove quick-pulls from the uprights and discard.** Attach uprights to the liner. Press firmly.
- F Fasten straps.** Fasten remaining straps 3 through 5.

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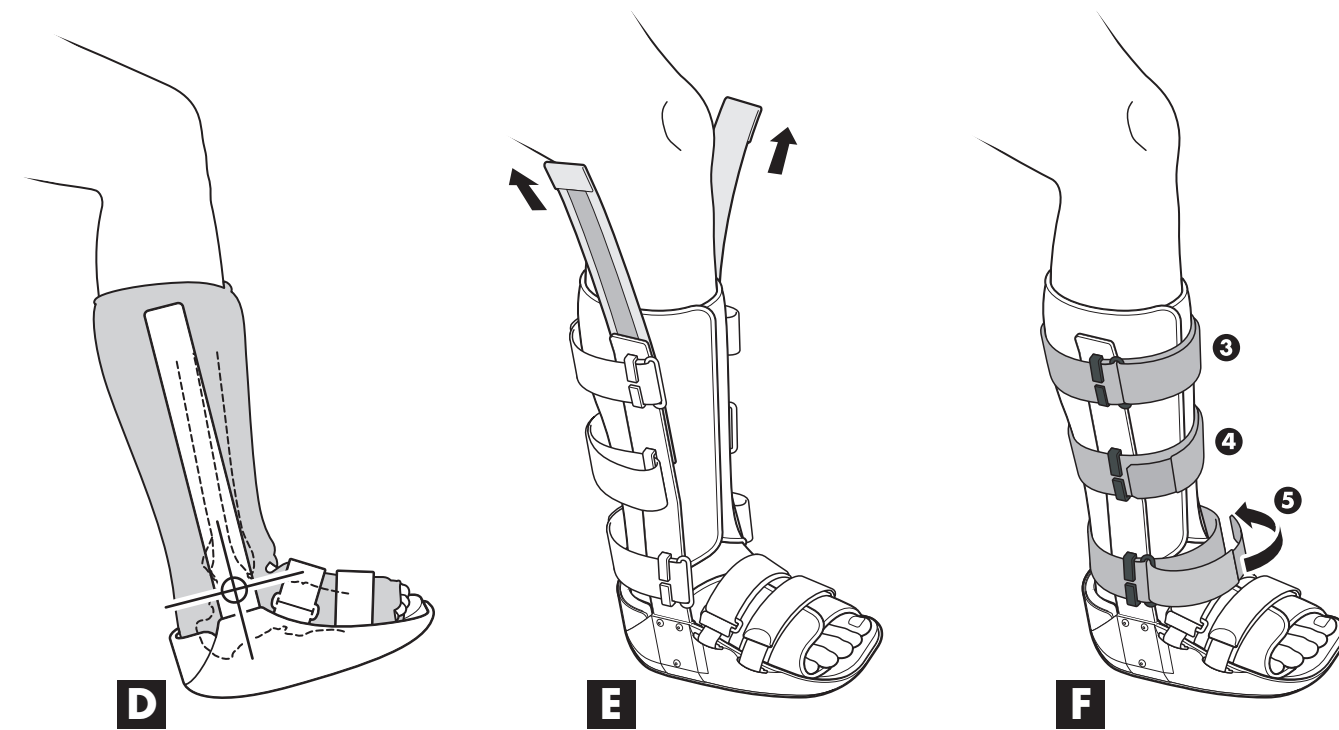
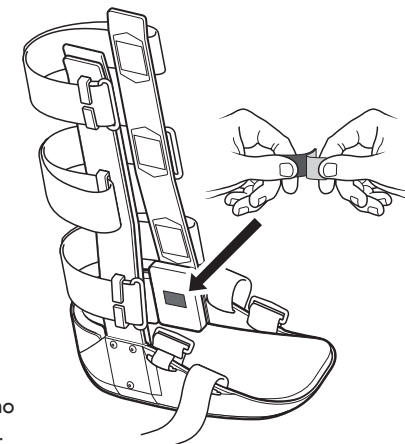
**Optional heel clip removal** Pull up on the heel clip.

**Optional heel clip replacement** Lift up foot bed padding and foot plate. Slide heel clip into place using the dove-tail slot guides. Release the foot plate and padding and push down into proper position.

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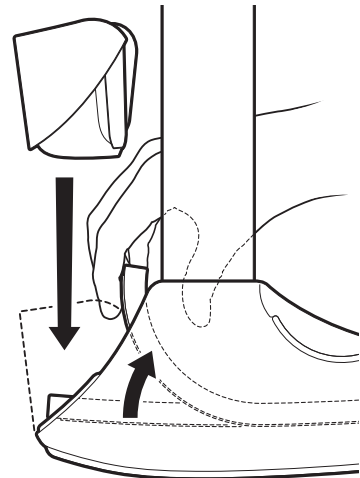
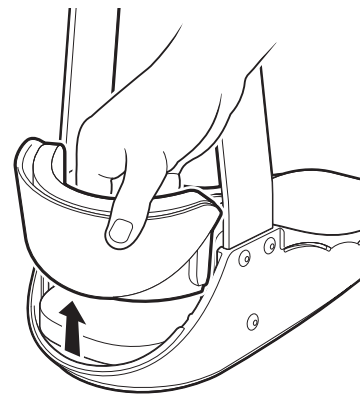
## USE AND CARE OF YOUR VECTRA:

- After initial application, the VECTRA may be removed and reapplied without removing liner from boot.
- Hand wash the liner and straps with mild soap, air dry. Do not place liner or straps into a mechanical dryer.
- Extra liners with strap sets are available from Customer Care: (800) 321-0607.

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