ENGLISH

INTENDED USE/INDICATIONS:

This brace is intended to provide stability and range or motion control following elbow injury or surgery.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- For single patient use only.
- If you experience increased pain, swelling, or any adverse
- Reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program.

INSTRUCTIONS FOR USE:

1. Position either the gel or air insert in the Tennis Elbow band/support.
2. With the strap loose, slide the band on to the forearm bringing the Insert into the desired position.
3. For Tennis Elbow position the insert below the elbow joint on the outside of the forearm.
4. For Golfers Elbow position the insert below the elbow joint on the inside of the forearm.
5. When the insert is properly positioned, adjust the strap for comfort without being too tight.

USE OF THE GEL INSERT FOR COLD THERAPY:

1. Cold therapy is recommended after injury or activity to reduce swelling.
2. Remove the gel insert from the band and place it in the freezer for a minimum of 1 hour.
3. Place the gel insert in the band and position it directly on the injured tendon area and adjust the strap for a comfortable fit.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Acetal, Ethylene Vinyl Acetate (EVA), Nylon, Polyvinyl chloride (PVC), Polyurethane

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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© 2025 Breg, Inc. All rights reserved.