ENGLISH

INTENDED USE/INDICATIONS:

OA knee bracing products are intended to provide relief for medial and lateral compartment osteoarthritis.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use.
- For single patient use only.
- If you experience increased pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also essential parts of a safe treatment program. Consult your licensed health care professional should you have any questions regarding the safe application of this product related to your medical history.
- Care and cleaning of this product is essential to its continuing strength and performance. Follow removal and cleaning process described below.
- When using braces bilaterally, modify activity as necessary to keep medial hinges from contacting each other.

INSTRUCTIONS FOR USE:

ADJUSTABLE HINGE INSTRUCTIONS

- 1. LOOSE PADS:** Turn upper/lower screws clockwise on medial/lateral hinges to move condyle pads inward for a tighter fit.
- 2. TIGHT PADS:** Turn screws counterclockwise to move condyle pads outward for a looser fit.
- 3. UNEVEN FIT (EDGES):** Tilt condyle pads to contour knee. Adjust upper/lower screws: clockwise for tighter edge, counterclockwise for looser edge.

COUNTERFORCE PLUS & X2K-OA INSTRUCTIONS

- A. LATERAL CONDYLE PAD:** Walk 5 minutes with brace. If more force is needed, turn screws clockwise to move pad inward. Adjust screws individually for comfort and fit.
- B. MEDIAL CONDYLE PAD:** After lateral pad is adjusted, turn screws clockwise for firm, comfortable contact with the knee. Note: If discomfort occurs, loosen screws or remove brace during inactivity.

LATERAL COMPARTMENT SYMPTOMS

Reverse medial compartment instructions. Attach gel pad to medial condyle basket for proper fit.

C. ADJUSTING FOR LEG GIRTH CHANGES

- 1. LOSS IN GIRTH:** Turn screws clockwise to tighten pads inward.
- 2. GAIN IN GIRTH (SWELLING):** Turn screws counterclockwise to loosen pads outward.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Aluminum, Neoprene, Nylon, Polyethylene (PE), Polyvinyl Chloride (PVC), Stainless Steel.

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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INSTRUCTIONS COUNTERFORCE PLUS & X2K-OA

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