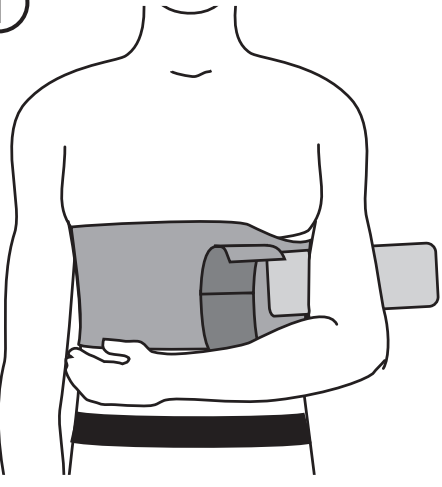
1

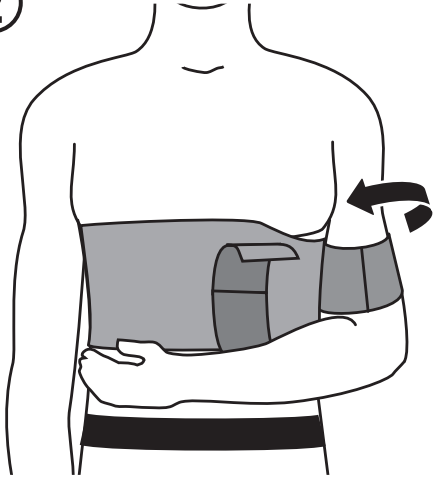
Secure immobilizer swathe around chest area with cutout portion under arm that is to be immobilized.

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2


Attach humeral cuff to immobilizing swathe with hook closure.

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Pasangkan manset humeral pada balutan pelumpuh dengan penutup cangkuk.

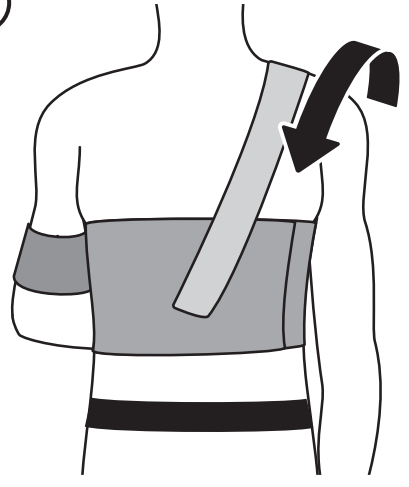3


**For Deluxe version:** Place adjustable shoulder strap over the unaffected shoulder, and secure to the back of the immobilizing swathe.

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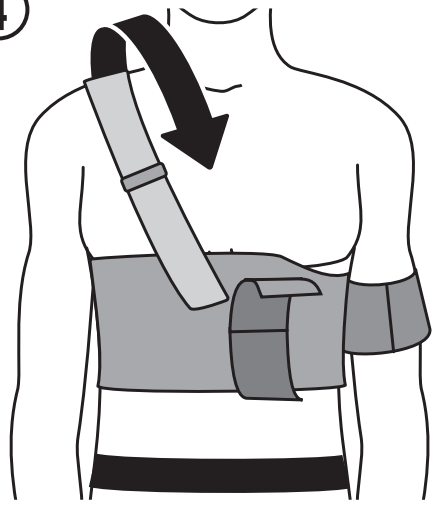4


**For Deluxe version:** Adjust shoulder strap for length and secure to front of the immobilizing swathe.

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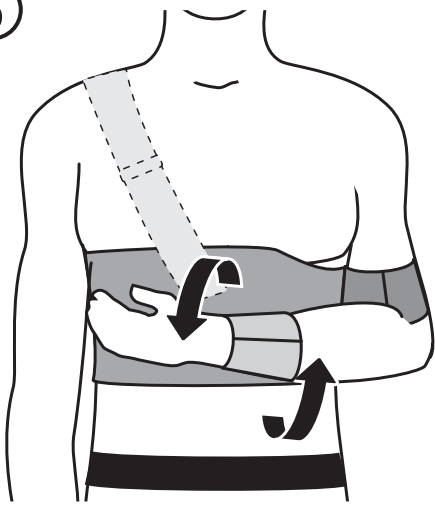5

Wrap and secure wrist/hand cuff to the immobilizing swathe.

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**BREG**

**Deluxe Straight Shoulder Immobilizer**  
**Straight Shoulder Immobilizer**

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AW-1.01931

## WARNINGS

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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© 2024 Breg, Inc. All rights reserved.