⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Deluxe Shoulder Slings and Immobilizers Instructions for Use

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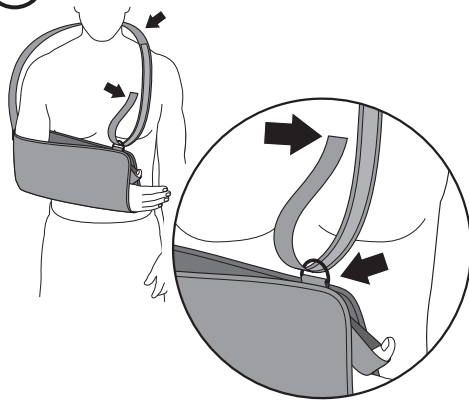
1

Wrap shoulder strap around neck and shoulders. Lace strap end through D-ring(s) and attach to shoulder strap.

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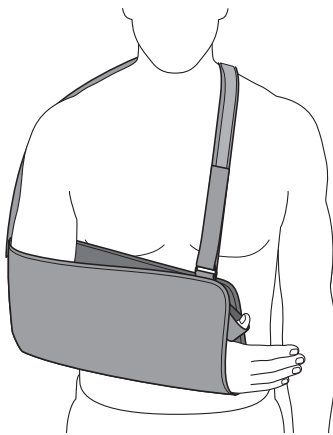
2

Properly applied sling with arm comfortably supported, and thumb in the thumb loop (if applicable).

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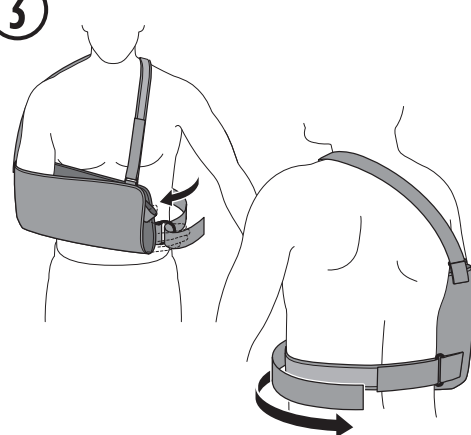
3

Shoulder Immobilizer: After fitting the sling as indicated above, wrap waist strap around waist and feed through plastic ring. Attach strap with hook and loop closure.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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© 2023 Breg, Inc. All rights reserved.