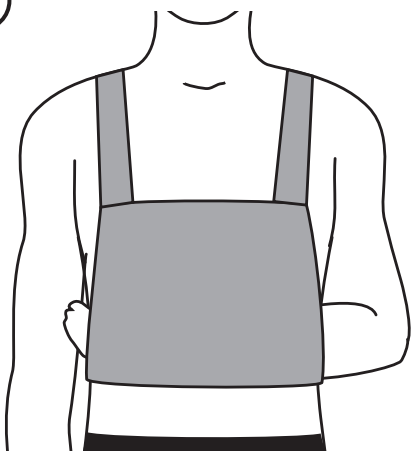
1

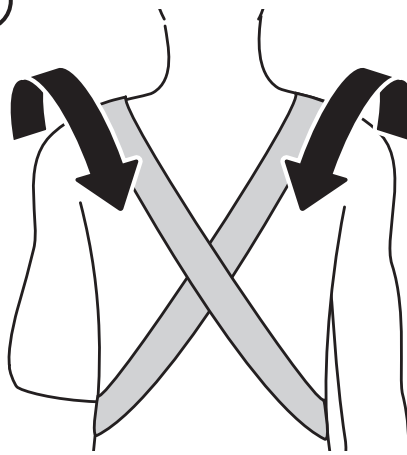
Position sling against rib cage. Bring web straps straight up and over the patient's shoulders. Place affected arm through the sling.

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2


Criss cross both straps against the patient's back.

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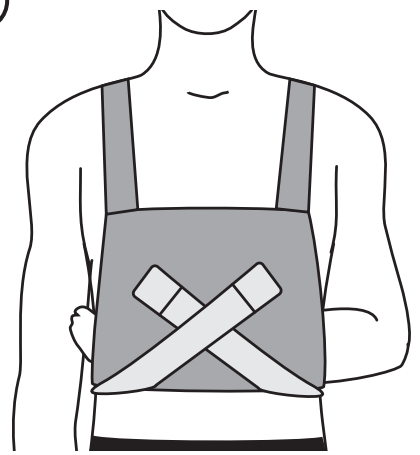3


Attach hook straps to the front of the sling in a criss cross motion supporting the affected arm and adjust to length.

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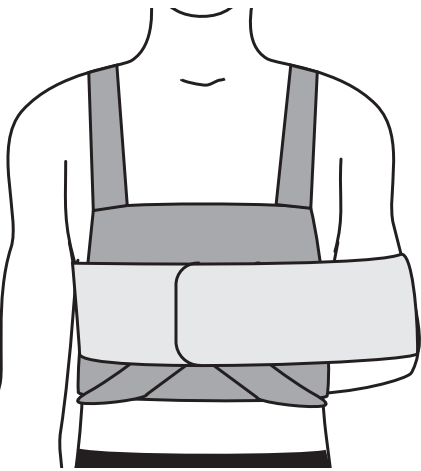4

Secure arm against the body by attaching the swathe strap to the sling. Place one end of the strap to the sling. Encircle the affected arm and bring around the patient's back and under the non-affected arm. Secure loose end to the front of the sling

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## Sling and Swathe Instructions for Use

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## WARNINGS

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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