
1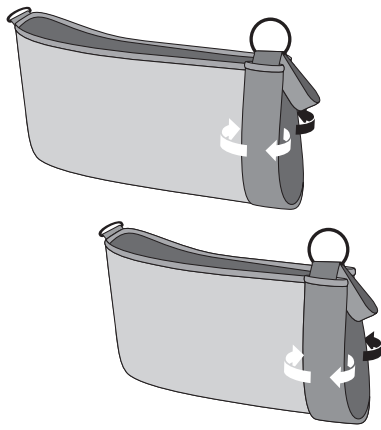


1. To adjust the size of the universal sling, roll the cuff at the front of the sling to the correct arm length. Slide arm into the sling.

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2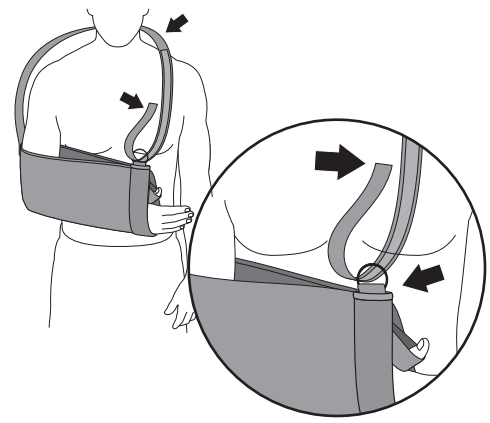


2. Wrap shoulder strap around neck and shoulders. Lace strap end through O-ring and attach to shoulder strap.

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3

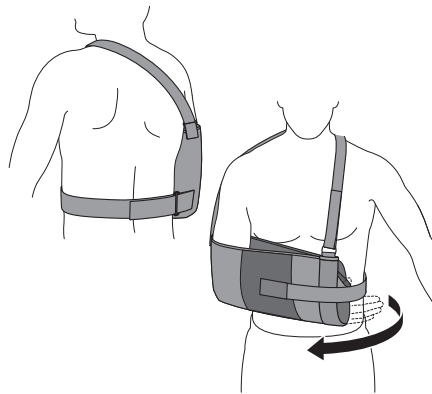3. Shoulder Immobilizer: After fitting the sling as indicated above, feed waist strap through plastic ring at the back. Wrap snugly around the waist and secure the strap to the hook and loop patch on the front of the sling.

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Universal Shoulder Slings and Immobilizers Instructions for Use

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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