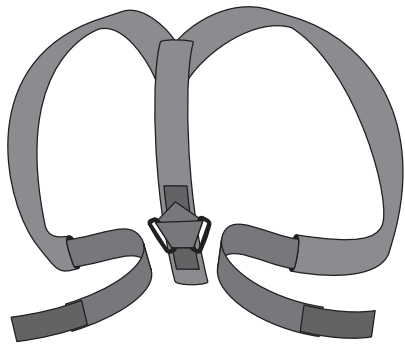
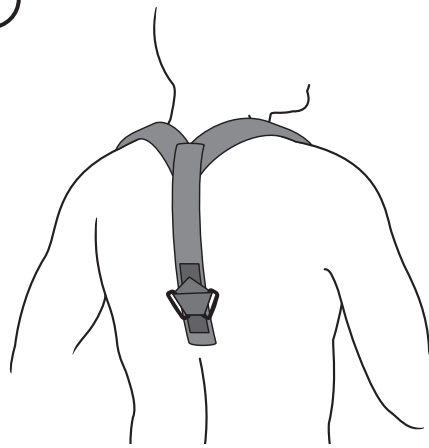
**1**

Unfasten hook and loop closure.

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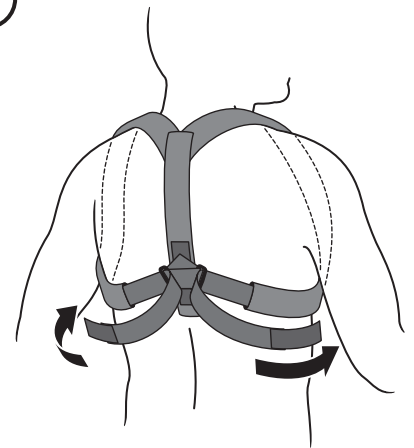
**2**

Position Clavicle Brace such that the center of the brace is centered on the patient's back. The D-rings should rest furthest from the patient's neck.

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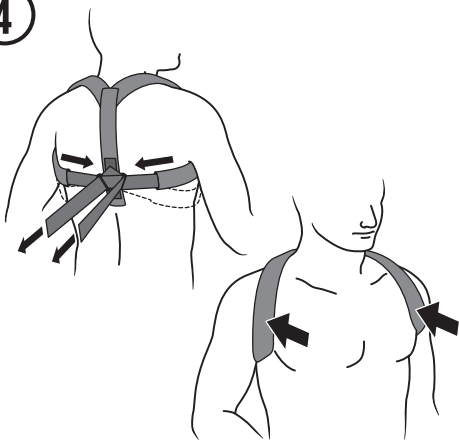
**3**

Position straps around the top of the patient's shoulders and wrap under the arm pits. Thread the side straps through the D-rings.

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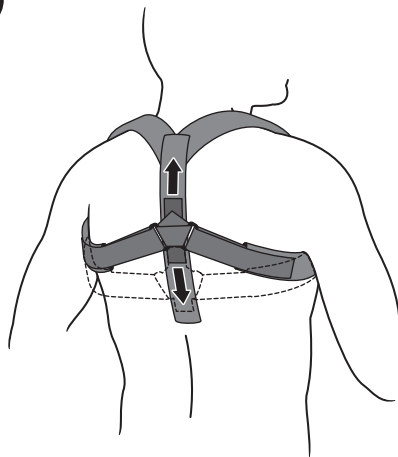
**4**

Tighten the straps, to pull the shoulders back, and fasten the hook tabs on the ends of the straps to the brace.

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**5**

The D-ring attachment site can be repositioned on the center strap for additional support and comfort.

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## Clavicle Support Instructions for Use

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## WARNINGS

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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Breg, Inc.