
1

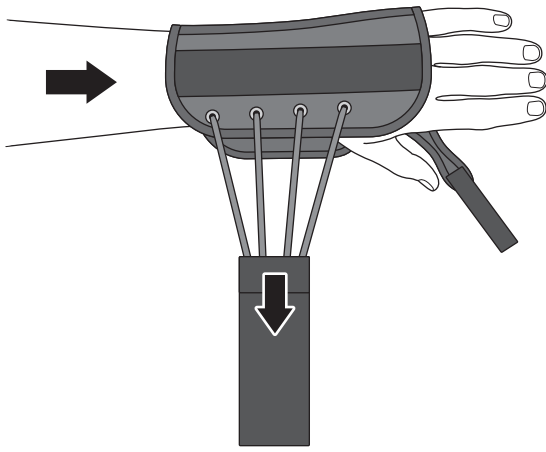
Slide hand into the brace and pull hook strap to tighten brace.

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2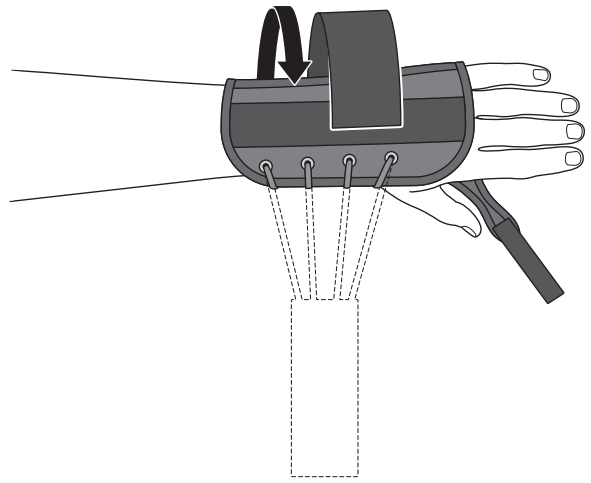


Wrap strap around the brace, and secure to the loop closure.

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3

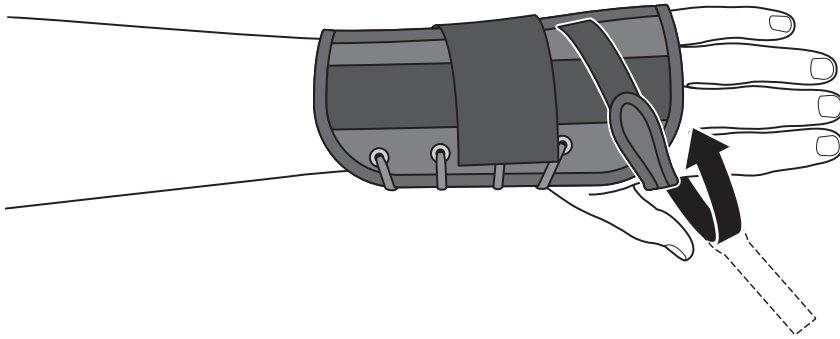Line up the palmar stay below the middle crease of the hand and secure the thumb strap. Adjust for comfort.

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Low Profile Wrist Instructions for Use

Example illustrates left hand brace application.

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Breg, Inc.