⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Patella Stabilizer Instructions for Use

Estabilizador de rótula Instrucciones de uso Stabilisateur de rotule Mode d'utilisation

2382 Faraday Avenue, Suite 300
Carlsbad, CA 92008 U.S.A.

Toll Free Tel: 1-800-321-0607

Local: +1-760-795-5440

Fax: +1-760-795-5295

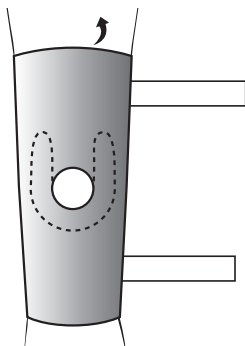
www.breg.com

© 2023 Breg, Inc. All rights reserved.

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

1**1. Apply to Knee**

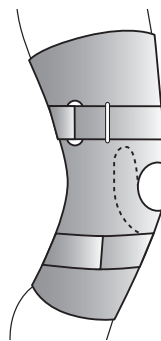
Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the knee so that the opening at the front surrounds the knee cap.

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2**2. Secure Straps**

Starting with the lower strap, feed straps through the D-rings and secure. Fit should be snug but not too tight so as to restrict circulation.

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NOTICE TO USERS

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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