⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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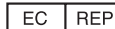
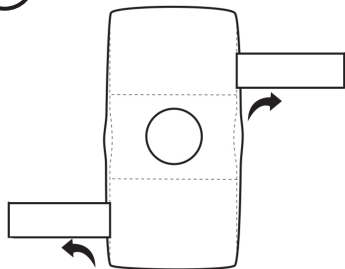
1

1. Loosen All Straps

Loosen all straps to their maximum circumference.

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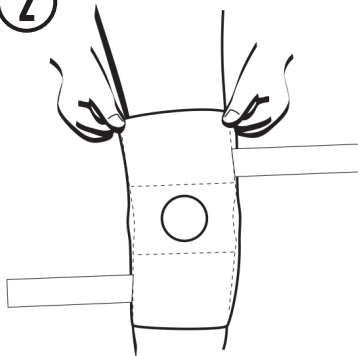
2

2. Apply Sleeve

Pull the sleeve onto the leg until the hinges line up with the center of the knee. Position the front hole directly over the knee cap.

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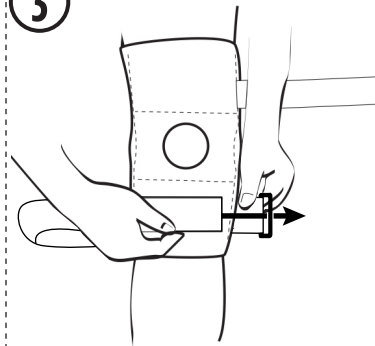
3

3. Secure Lower Strap

Wrap lower strap around the brace, then loop strap through D-ring. Pull strap back to tighten and press tab end to secure.

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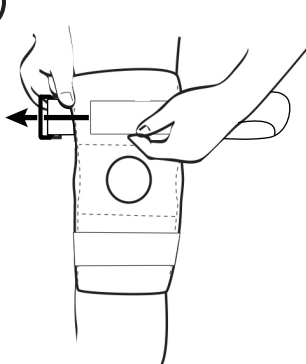
4

4. Secure Upper Strap

Wrap upper strap around the brace, then loop strap through D-ring. Pull strap back to tighten and press end to secure.

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Material: Neoprene or AirMesh®

CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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© 2017 Breg, Inc. All rights reserved.