⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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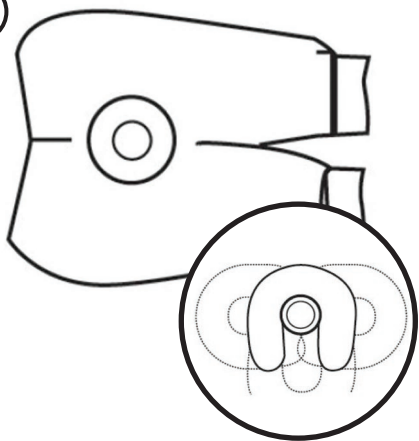
1


Open product so flaps are free. Position "horseshoe" buttress as desired.

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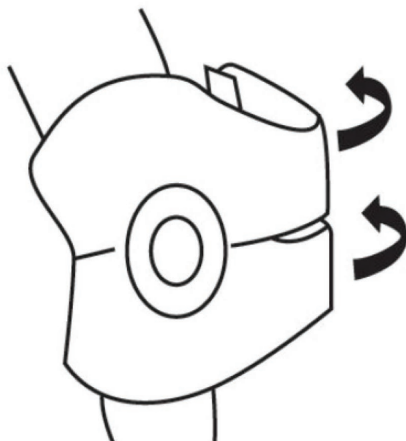2

Position brace so that the patellar opening surrounds the knee cap.

Coloque la rodillera de manera que la abertura de la rótula rodee la rótula de la rodilla.

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3

Holding brace in place with one hand, reach around with the other hand and secure the bottom strap, then the top strap. Readjust both straps as necessary to achieve a snug fit.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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Breg, Inc.