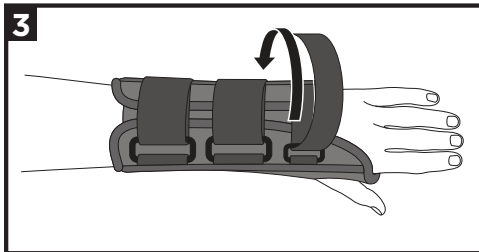
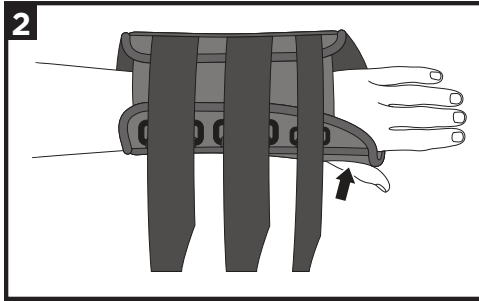
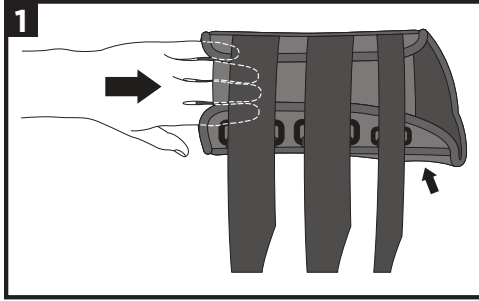
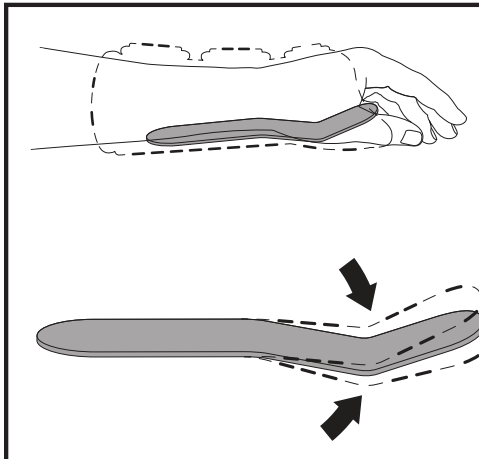
CLASSIC WRIST BRACE

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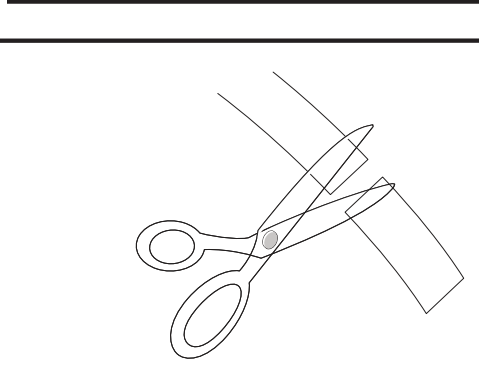
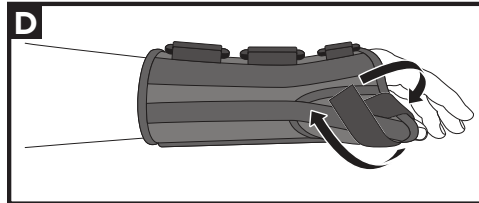
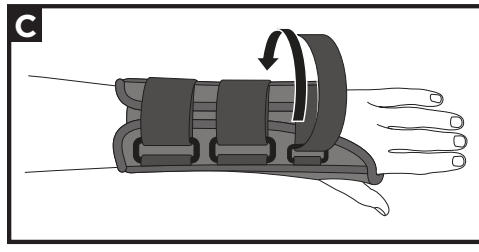
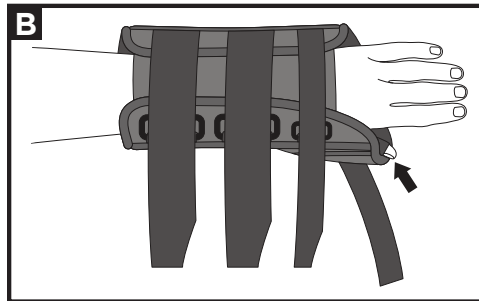
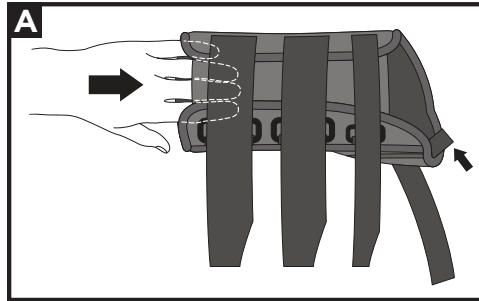
ENGLISH

INTENDED USE/INDICATIONS:

This brace is intended to provide support and stabilization of the wrist joint and forearm where indicated by a healthcare professional.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Follow application and care process described in the instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

CLASSIC WRIST BRACE

1. Slide the brace onto affected hand. Be sure thumb and thumb hole on the side of the brace are aligned.
2. Slide thumb into the hole in the side of the brace.
3. Thread the three closure straps through D-rings and secure to brace.

CLASSIC WRIST BRACE W/ THUMB SPICA

- A. Slide the brace onto affected hand. Be sure thumb and elastic thumb loop are aligned.
- B. Slide thumb into elastic thumb loop.
- C. Thread the three closure straps through D-rings and secure to brace.
- D. Wrap thumb strap around the injured thumb and secure to brace.

BRACE MODIFICATION/ADJUSTABILITY

- The curvature of the metal stay on the bottom of the brace should fit into the palm of the hand and extend down the forearm.
- If necessary, remove the malleable metal stay and bend it to the appropriate position.
- Trim hook and loop closure straps, leaving approximately 1" - 2" of overlap.

USE AND CARE:

Remove metal stay. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Aluminum, Cotton, Nylon, Polyester

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

AW-1.03603 Rev C 04/25

© 2025 Breg, Inc. All rights reserved.

www.breg.com

Local: +1-760-795-5440

Fax: +1-760-795-5295

Toll Free Tel: 1-800-321-0607

Carlsbad, CA 92008 USA

2382 Faraday Avenue, Suite 300

breg.com/classicwrist

Classic Wrist Brace

Example illustrates left hand brace application.

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