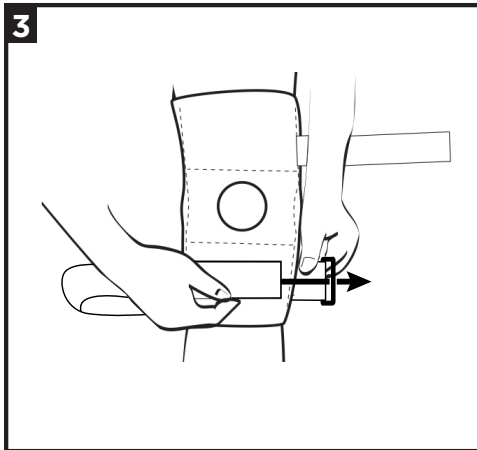
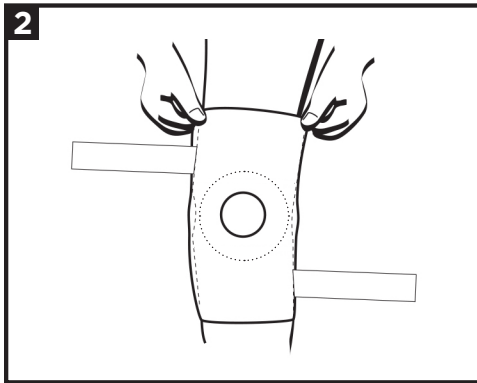
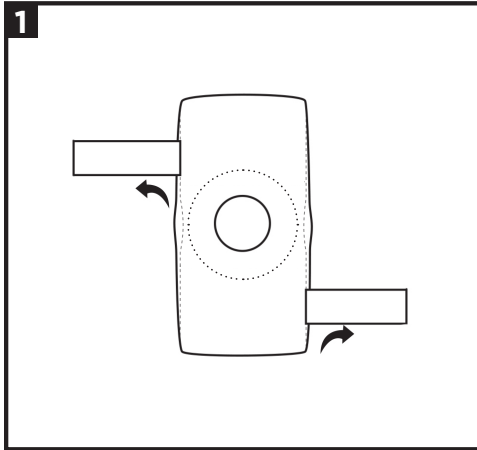
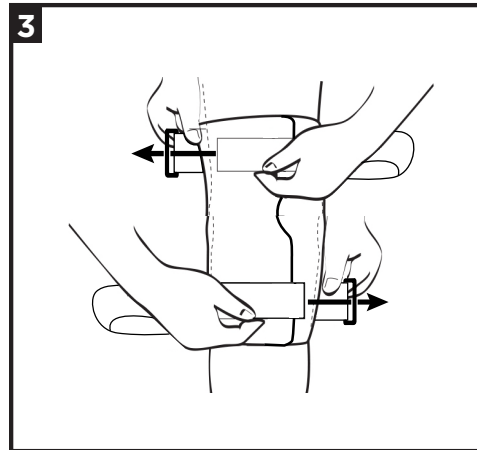
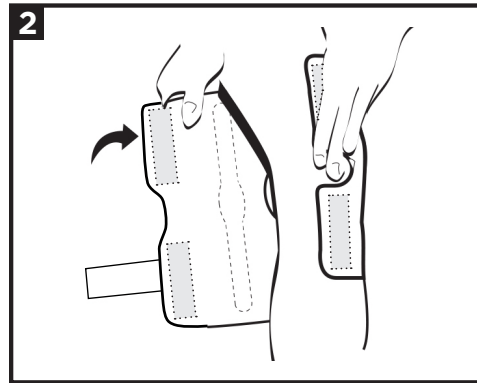
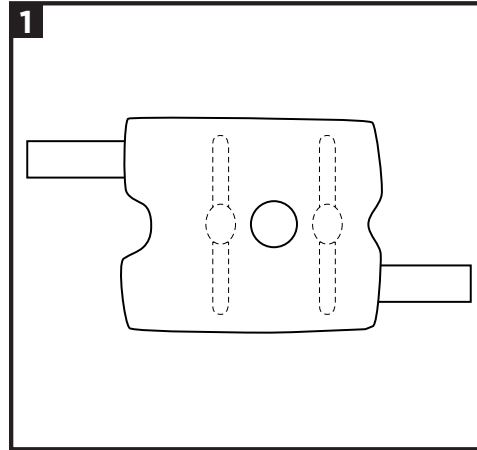
PULL ON
COLOCACIÓN
TIREZ

WRAP AROUND
ENROLLAR ALREDEDOR
ENROULEZ AUTOUR

ENGLISH
INTENDED USE/INDICATIONS:

Soft Knee Bracing products provide control and support for ligament injuries and instabilities.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- If swelling increases or pain, numbness or skin irritation occurs, discontinue use and contact your medical provider immediately. This product is only a support device, not to prevent injuries or prevent recurrence of injuries.
- Follow application and care process described in instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

1. Unfasten straps.
2. Step into the brace at the widest opening. Pull the brace up the leg until hinges align with the knee. For the wrap around version, align the hinges on the sides of the knee. Wrap the brace around the leg and secure the flaps. The opening should be directly over the knee cap.
3. Loop straps through the rings and pull back over the top of the lower thigh and upper calf, then fasten the hook and loop closure.
4. Adjust straps for comfort and fit. The brace should be snug, but not uncomfortably tight.

NOTE: Do not overtighten, adjust as necessary.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Aluminum, Neoprene, Nylon, Polyoxymethylene (POM)

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

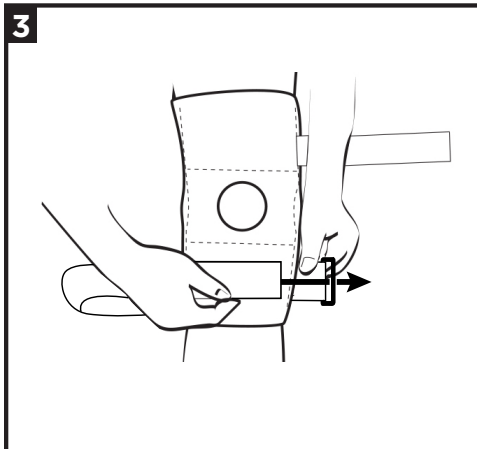
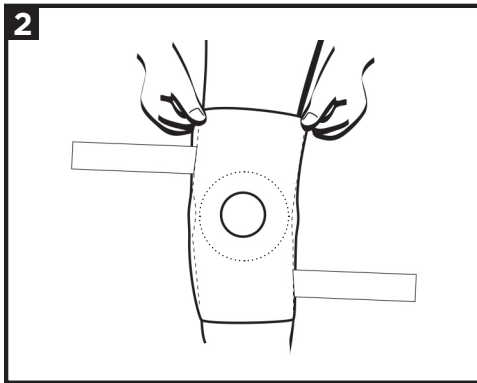
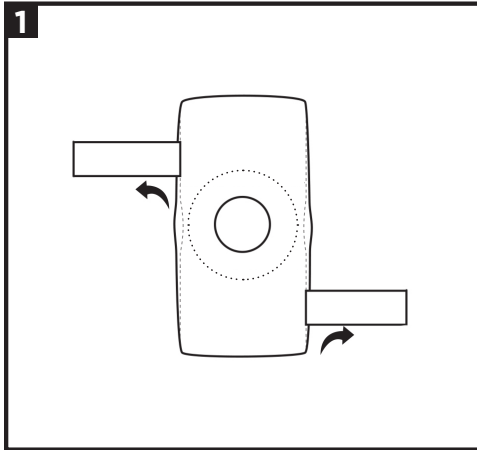
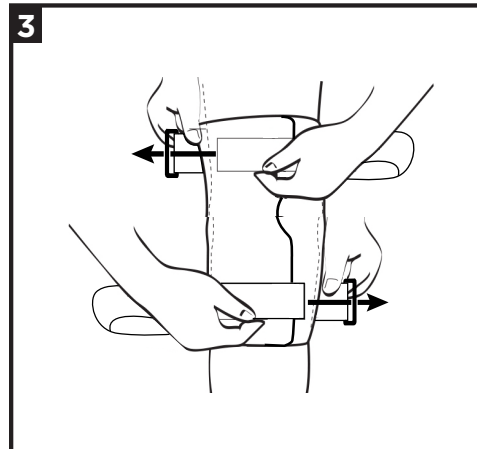
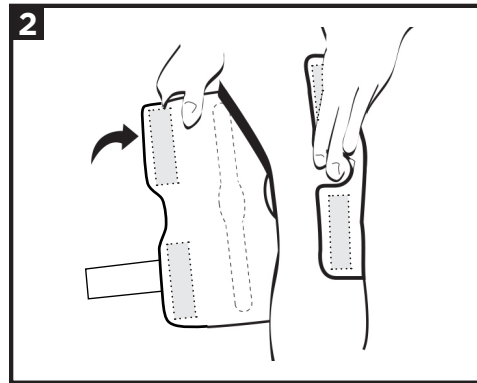
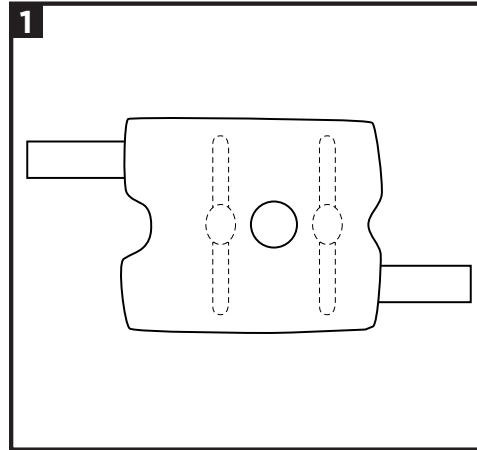
AW-1.03604 Rev C 04/25
© 2025 Breg, Inc. All rights reserved.
www.breg.com
Local: +1-760-795-5440
Fax: +1-760-795-5295
Toll Free Tel: 1-800-321-0607
Carlsbad, CA 92008 USA
2382 Faraday Avenue, Suite 300

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