## ⚠️ WARNINGS ⚠️

- WARNING:** Remove before walking. Walking while wearing the Dorsal Night Splint could result in slipping and falling.
- WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- WARNING:** If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your medical professional regarding safe and appropriate activity level while wearing this device.
- CAUTION:** Follow application and care process described in instructions.
- CAUTION:** For single patient use only.

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# Adjustable Night Splint Instructions for Use

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1

Remove the Dorsal Night Splint from bag and open hook closures.

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2

Slide foot and heel into elastic piece and tighten the ankle straps and the toe straps. Readjust if necessary for comfort.

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3

Wrap the calf strap around the calf, and adjust for comfort if necessary.

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**CARE & WASHING INSTRUCTIONS**

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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© 2019 Breg, Inc. All rights reserved.
Breg, Inc.