⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your medical professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Lace-Up Ankle Brace Instructions for Use

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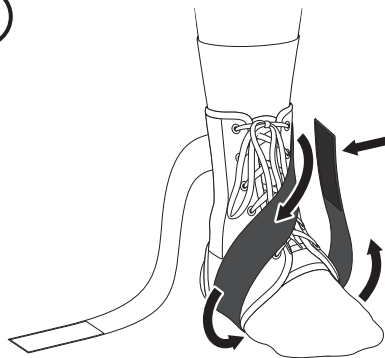
1

The Lace-Up Ankle Brace is designed to be worn over an athletic sock. Position heel in the hole at the rear and bottom of the brace. Lace the ankle brace and tie snugly.

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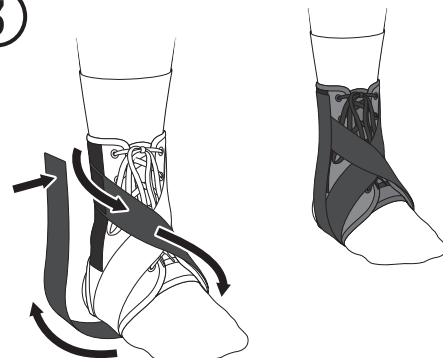
2

Wrap the inside (medial) strap across the top of the foot. Continue by wrapping the strap under the foot and heel. Secure the strap to the inside of the ankle. Pull strap tight to ensure proper performance.

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3

Wrap the outside (lateral) strap across the top of the foot. Continue by wrapping the strap under the foot and heel. Secure the strap to the outside of the ankle.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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© 2019 Breg, Inc. All rights reserved.
Breg, Inc.