# Latitude™ OA Knee Brace

- Latitude™ OA-Knieorthese
- Genouillère pour gonarthrose Latitude™
- Rodillera para osteoartritis Latitude™
- Longherone per l'osteoartrite Latitude™
- Latitude™ OA knestøtte
- Latitude™-knæledsørtose
- Latitude™ OA Knäortos
- Latitude™ OA -polvituki nivelrikon hoitoon
- Latitude™ OA-kniebrace
- Joelheira para osteoartrite Latitude™
- Latitude™ OA 膝蓋支架
- Orteza stawu kolanowego Latitude™ do stosowania w chorobie zwyrodnieniowej stawów
- Pendakap Lutut Osteoartritis Latitude™

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



**ENGLISH**  
**INTENDED USE / INDICATIONS**  
 OA knee bracing products are intended to provide relief for medial and lateral compartment Osteoarthritis.

**INTENDED USER**  
 The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**KONTRAIKATIONER**  
 Do not use if you are allergic to any of the materials contained within the product.



**⚠ WARNINGS & PRECAUTIONS ⚠**

- For single patient use only.
- The brace is not designed to provide support for previous ligament injuries.
- If you experience increased pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed healthcare professional regarding safe and appropriate activity level while wearing this device.
- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.
- Use in high impact activities may cause premature wear of components.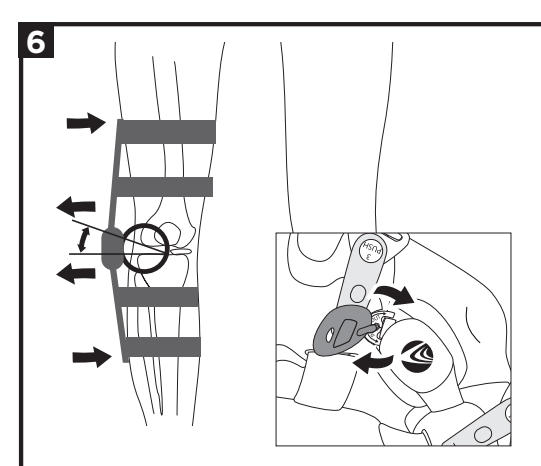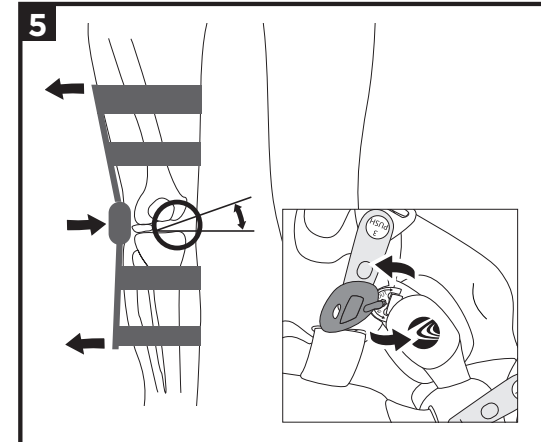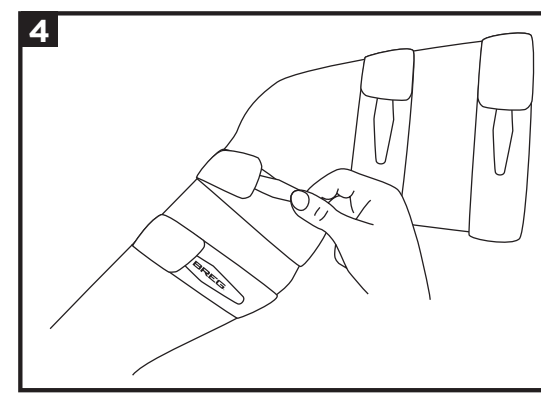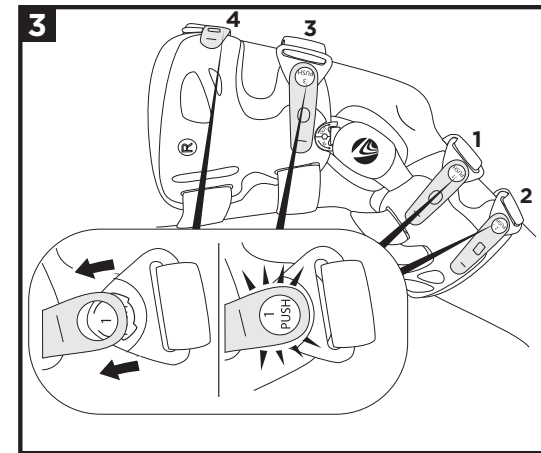

R = Right and	L = Vasen
L = Left	R = Rechts,
R = Rechts und	L = Links
L = Links	R = Direita e
R = Droite et	L = Esquerda
L = Gauche	R = 右, L = 左
R = Derecha,	R = 우측
L = Izquierda	L = 좌측
R = Destro e	R = strona prawa i
L = Sinistro	R = strona lewa
R = Høyre og	R = Kanan dan
L = Venstre	L = Kiri
R = Højer och	
L = Vänster	
R = Oikea,	

Key	Nakkel	Chave
Schlüssel	Nagle	钥匙
Clé	Nyckel	키
Llave	Avain	Kluczyc
Chiave	Sleutel	Kunci

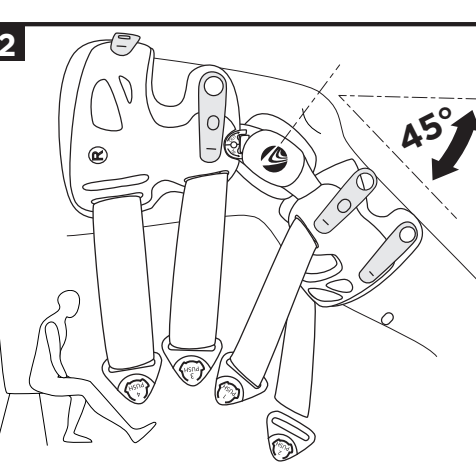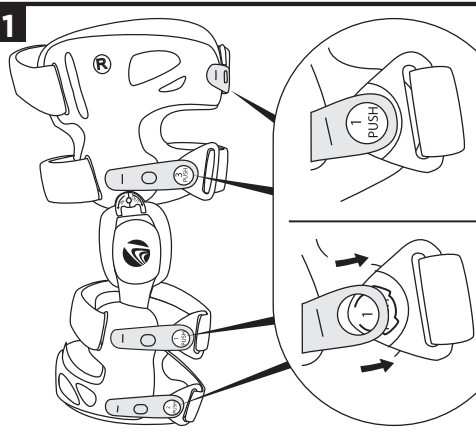**INSTRUCTIONS FOR USE**  
**INITIAL FITTING PROCESS**

- Push the release button that says "PUSH" and slide to detach strap. Repeat for all four straps.
- Sit on a stable surface with knee bent at approximately 45 degrees. Position brace on the lateral (outside) of the leg. Align the center of the hinge to slightly above the patella (knee cap).
- Reattach the straps in sequence (1-4) by sliding clip into place.
- For final brace adjustment, verify proper hinge alignment and adjust strap tension as needed.

**MEDIAL / LATERAL OFFLOADING**

- Medial offloading:** For medial compartment offloading, turn key counterclockwise ("IN"). Drawing 5 shows a right leg with medial offloading.
- Lateral offloading:** For lateral offloading, turn key clockwise ("OUT"). Drawing 6 shows a right leg with lateral offloading.

**BRACE COMFORT ASSESSMENT**  
 Once the brace is applied, stand and walk a short distance to ensure comfort and proper correction. If there is any discomfort, adjust the straps as needed. If you experience pain in the knee joint or leg, turn the hinge adjustment back slightly. If more relief is required, turn the hinge adjustment no more than ¼ of a turn, as directed by your healthcare professional.

**USE AND CARE**  
 Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry. To clean the brace frame and hinge, wipe with a damp cloth. Avoid brace contact with salt water, sand, dirt and chlorinated water. Use a dry silicone spray if a hinge lubricant is needed.

**MATERIAL CONTENT**  
 Aluminum, Ethylene-vinyl Acetate (EVA) foam, Nylon, Polyester, Polypropylene, Polyurethane, Stainless Steel

**WARRANTY**  
 Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com for additional information.

**DISPOSAL:**  
 Dispose of product according to local regulations.

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**NOTICE TO USERS:** If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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