
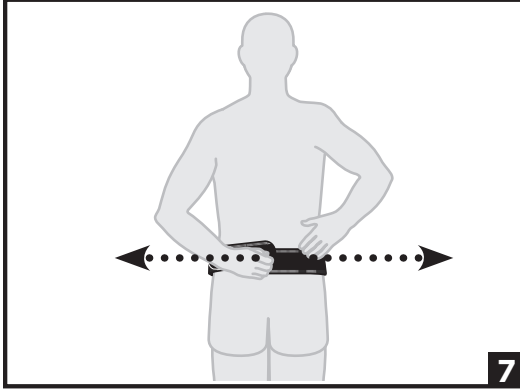
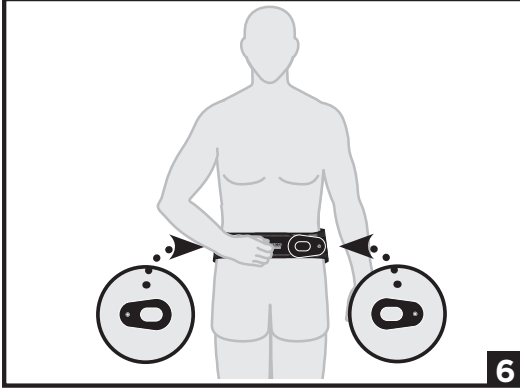
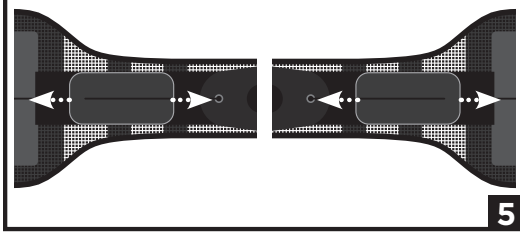
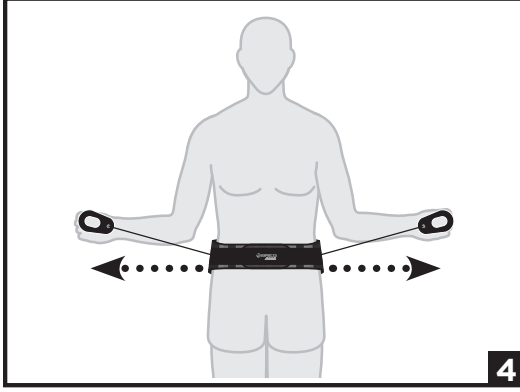
Size	Waist Size
Small	25" – 33" (64–84cm)
Medium	33" – 49" (84–124cm)
Large	41" – 65" (104–165cm)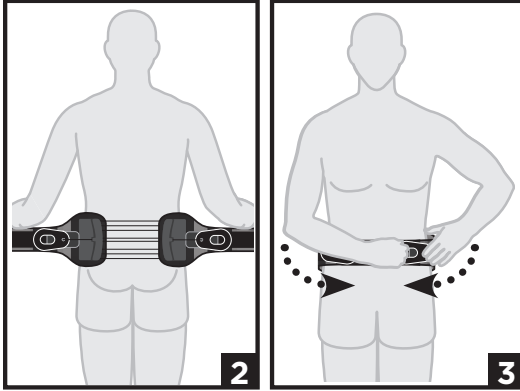

ENGLISH

INTENDED USE/INDICATIONS:

The intended use of these Lumbar products is to provide stabilization and motion restriction for the treatment of trauma related injury, acute post-surgical recovery, and the management of chronic conditions of the spine. Indicated for SI joint dysfunction, low back sprains / strains, sacroiliac joint fusion, symphysis pubis dysfunction, piriformis syndrome, sacroiliitis, arthritis, sciatica, injections.

INTENDED USERS:

The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
- The frequency and duration of use shall be prescribed by a healthcare professional.
- For single patient use only.

INSTRUCTIONS FOR USE:

HOW TO APPLY THE BRACE

- Measure Waist:** Use the sizing chart to select the correct size.
- Position Brace:** Center the lacing system on the spine, over the sacroiliac joint.
- Wrap Belt:** Pull the right belt over the left and fasten.
- Tighten:** Pull the handles outward to adjust tension, then attach them to the belt.
- Initial Setup:** Detach cord control tabs from handles and place them on the belt as preferred. This is only needed once.

HOW TO REMOVE THE BRACE

- Detach the pull handles and place them near the back of the brace.
- Unfasten the front hook-and-loop closure.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Airmesh®, Cotton, Nylon, Silicone, Polyester

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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MATÉRIAUX :

Airmesh®, coton, nylon, silicone, polyester

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NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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