ASCEND™ SI 621 LUMBAR ORTHOSIS

SMALL #SP10621-020

MEDIUM #SP10621-030

LARGE #SP10621-040

INSTRUCTIONS FOR USE

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ENGLISH

MEASUREMENT






1. Measure patient's waist and refer to sizing chart (**above**) for corresponding size.

APPLYING THE BRACE

NOTE: Before applying the brace, make sure the pull handles are positioned close to the lacer system.

2. Align the center of the lacer system to the patient's spine.
3. The belt should be positioned on the lower back, across the sacroiliac joint.
4. Fasten the brace around the patient's waist by pulling the right belt over the left belt.
5. Adjust the tension of the brace by pulling the pull handles away from the body, then attach the pull handles to the belt.
6. During initial fitting, detach the cord control tabs from the pull handles and position on the belt based on patient preference. These tabs only need to be positioned during the initial fitting and they remain in place thereafter.

Size	Waist Size
Small	25" – 33" (64 – 84cm)
Medium	33" – 49" (84 – 124cm)
Large	41" – 65" (104 – 165cm)

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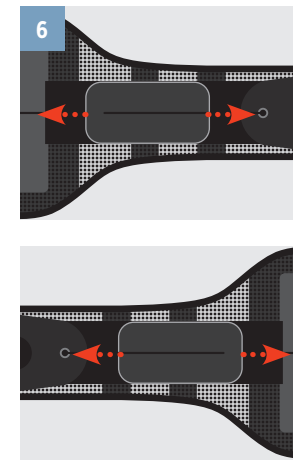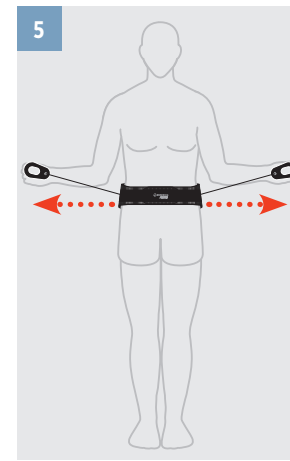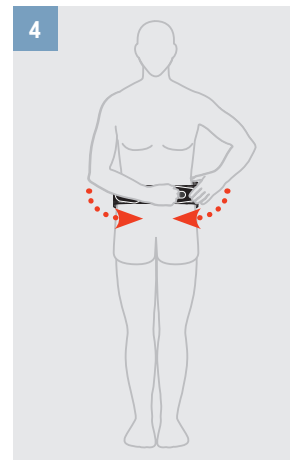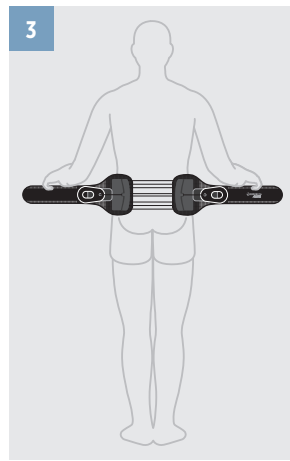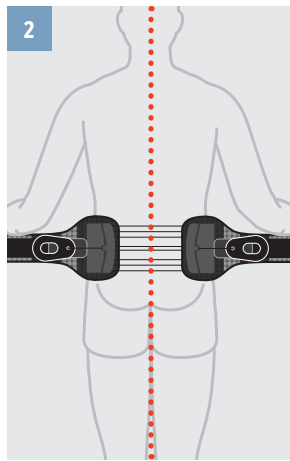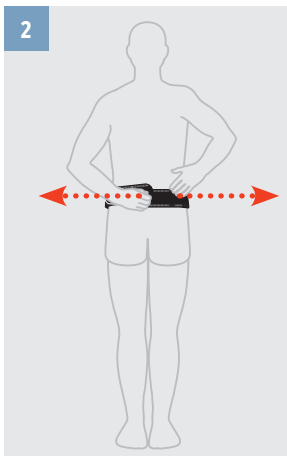
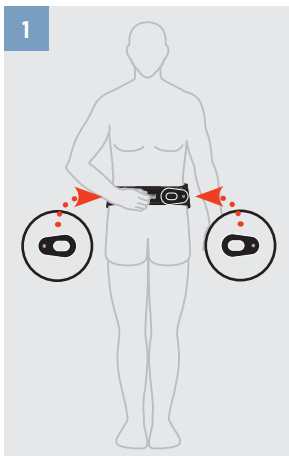
ENGLISH

REMOVING THE BRACE

1. Detach the pull handles from the belt and position them close to the back of the brace.
2. Detach hook and loop at the front to remove the brace.

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: This device is offered for sale by or on the order of a licensed health care professional.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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©2023 Breg, Inc. All rights reserved.
Ascend is a trademark of Breg, Inc.