# ASCEND™ CERVICAL PEDIATRIC COLLAR

MODEL #SP30272-010

## INSTRUCTIONS FOR USE

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[www.breg.com](http://www.breg.com)

©2021 Breg, Inc. All rights reserved.  
Ascend is a trademark of Breg, Inc.

AW-1.03818 Rev B 07/21

E/U authorized representative  
MDSS GmbH, Schiffgraben 41  
D-30175 Hannover, Germany

## SIZING

### PD 1

1-18 MONTHS

Weight: 11-29 lbs  
(5-13 kg)

Length: 21-33 in  
(53-84 cm)

### PD 2

9-24 MONTHS

Weight: 22-33 lbs  
(10-15 kg)

Length: 29-37 in  
(74-94 cm)

### PD 3

1-3 YEARS

Weight: 24-36 lbs  
(11-16 kg)

Length: 33-40 in  
(84-102 cm)

### PD 4

2-5 YEARS

Weight: 26-42 lbs  
(12-19 kg)

Length: 35-45 in  
(89-114 cm)




### PD 5

3-6 YEARS

Weight: 27-54 lbs  
(12-25 kg)

Length: 37-48 in  
(94-122 cm)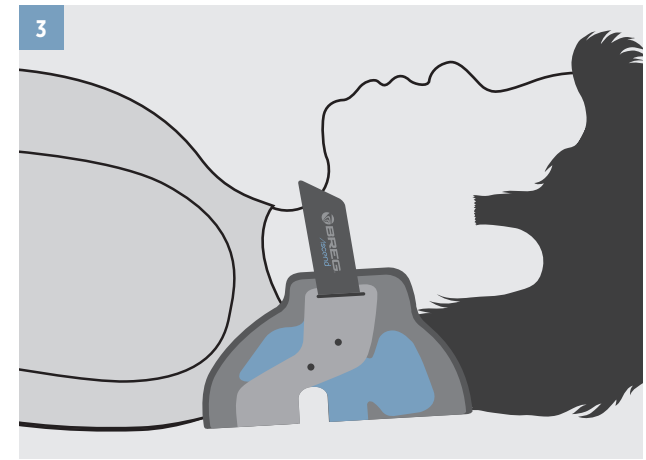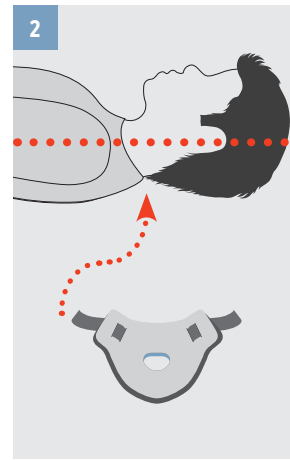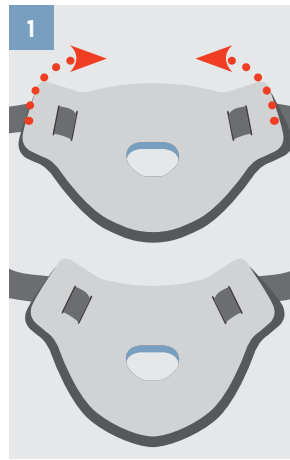

## ENGLISH

**SIZING NOTE:** The age, weight, and height ranges are for reference only. Always ensure the collar is supporting the chin when the patient's head is in a neutral position, the flexible tabs are flexed, and the collar is not contacting the tracheal area.

### BACK PANEL POSITIONING

1. Bend ends of the back panel inward to conform to the shape of the neck.
2. Ensure patient's head is in a neutral position and slide back panel under patient's neck.
3. Ensure loop straps are positioned between patient's ears and shoulders. The back panel should be centered along the midline of the back of the neck.

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ENGLISH

#### FRONT COLLAR POSITIONING

4. Bend the ends of the front collar inward to conform to the shape of the neck.
5. Place the bottom of front collar against the patient's chest and align sides with back panel loop straps.
6. Attach straps to the hook on the sides of the front collar.
7. Ensure both straps have an equal amount of tension and proper alignment is maintained, resulting in a snug and supported fit.

**NOTE:** Ensure that chin piece is not pressing on patient's throat. If it is, adjust side panels of the front collar so they are off the shoulders.

**Once fitted, ensure that the height of the front collar is supporting the chin when the patient's head is in a neutral position. If not, confirm sizing is correct.**

**NOTE:** Excess strap material can be trimmed to desired length.

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 WARNINGS 

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the collar, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** This device is offered for sale by or on the order of a licensed health care professional.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

**CARE & WASHING INSTRUCTIONS**

Hand wash pads in cold water with a mild detergent, rinse and air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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