



EXTRA TALL

TALL

REGULAR

SHORT

BREG ASCEND[®] COLLAR


PROPER FITTING STEPS

1. The chin should rest comfortably in a neutral position on the chin piece.
2. The front and back tabs should bend slightly.
3. The collar should not contact the tracheal area and should fit snug around the user's neck.

HOW TO USE THE SIZING GUIDE

Place the card on the subject's shoulder and hold it straight against the subject's head as shown in the image to the right.

Draw an imaginary line from the bottom of the chin to the color bars on the sizing guide to identify the corresponding size Breg Cervical Collar.

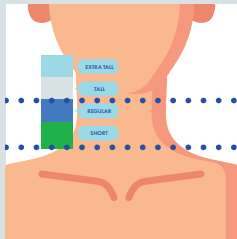

This sizing guide and its instructions are not intended as medical advice. Prior to sizing, the user should review the Breg Instructions for Use and receive applicable training.

BREG PEDIATRIC COLLAR SIZING OPTIONS

PROPER FITTING STEPS

1. The chin should rest comfortably in a neutral position on the chin piece.
2. The front and back tabs should bend slightly.
3. The collar should not contact the tracheal area and should fit snug around the user's neck.

PEDIATRIC SIZE OPTIONS



PD 1

1-18 MONTHS

Weight: 11-29 lbs
(5-13 kg)

Length: 21-33 in
(53-84 cm)



PD 2

9-24 MONTHS

Weight: 22-33 lbs
(10-15 kg)

Length: 29-37 in
(74-94 cm)



PD 3

1-3 YEARS

Weight: 24-36 lbs
(11-16 kg)

Length: 33-40 in
(84-102 cm)



PD 4

2-5 YEARS

Weight: 26-42 lbs
(12-19 kg)

Length: 35-45 in
(89-114 cm)



PD 5

3-6 YEARS

Weight: 27-54 lbs
(12-25 kg)

Length: 37-48 in
(94-122 cm)

The sizing options are not intended as medical advice. Prior to sizing, the user should review the Breg Instructions for Use and receive applicable training.

